## LEMI 4 / LEMI 4 BI-ZAK / LEMI SYNCRO BI-ZAK / LEMI MED

ORIGINAL INSTRUCTIONS - REV. 5 / 2023

LEMI 4

LEMI MED

LEMI 4 BI-ZAK /  
LEMI SYNCRO BI-ZAK

INSTRUCTION FOR USE ..... pag. 3

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NOTICE D'UTILISATION ..... pag. 31

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**EN**

Congratulation for having chosen an equipment LEMI produced by Brusaferrri & C.

This chair has been designed and built using state of the art technologies and processes to guarantee the best reliability in the time and ensure maximum safety.

For a correct and safe operation of the chair is essential to carefully read and observe what is indicated in this manual. All documentation accompanying the chair – and in particular this manual – must be carefully preserved for future reference.

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## Symbols meaning

Non-compliance of this procedure could cause damages to the chair or its parts.

Non-compliance of this procedure could cause health damages or injury to the operator or people near the chair

# 1. INTRODUCTION

The successful and safe operation of the chair is guaranteed only if it is used in accordance with the specifications in this manual and, in general, in the accompanying documentation; therefore it is essential to carefully read and keep all the relative documentation. Although the chair is equipped with active and passive safety features, not all risks can be avoided due to its incorrect use. You must always make sure that all users have fully understood the rules of use; the manufacturer declines any responsibility for damage to persons or property resulting from improper use of the device. Do not remove or damage the labels, notices and warnings placed on side of the chair. In case you need to restore them, please contact the manufacturer. The manufacturer Brusaferrì & C., declines any and every responsibility for failure to observe the rules of safety and prevention described in the various sections of this manual and for any damage caused by improper use. All interventions on the chair (maintenance, adjustments, repairs, cleaning) must be carried out by suitably trained personnel and in accordance with what is indicated in this manual.

## 1.1. Destination of use and lifetime

The chair LEMI 4 - LEMI 4 BI-ZAK - LEMI MED has been designed and built for the execution of face / body beauty treatments, pressotherapy, manicures, pedicures, diagnostic visits, blood samples, dermatology and small cosmetic surgery interventions, chemotherapy, plasmapheresis and rehabilitation.

**The estimated service life is 10 years.**

**WARNING! It is forbidden any use of the chair other than that described in this manual. Brusaferrì & C. declines any responsibility for malfunctions or damages to persons or property due to uses other than those specified in this manual.**

**The chair should be used only by personnel who are informed about the use mode, operation and its safety equipment.**

## 1.2. Preservation of the manual

This operating manual should be kept for future reference. We recommend:

Keep the manual in a place accessible and known to all operators, protected from moisture and heat and protected from the direct sun beams. Use the manual in order not to damage any part of the content: do not remove, tear or modify any part of the manual.

In the case of sale or transfer of the chair to another person, this manual and its annexes should be delivered intact to the new users.

**WARNING! Please read this operation manual before using the chair. Anyone who uses the chair should be properly informed on the parts of this manual relevant to the operations to be performed.**

## 1.3. Labelling of the chair

The chair is provided with labels with the following data, in conformity with actual rules (see § 14. Label Legend)

**WARNING! It is recommended not to remove the labels on the machine; labels must be preserved well fixed, whole and easy to read.**

**WARNING! The data present on each label, must be reported in case of need of assistance or spare parts.**

## 1.4. Warning messages

The chair is provided with signs showing phrases or symbols; pay attention to the warnings on the equipment.

The symbol that warns against the risk of electrocution is located next to the plug of the electric supply.

The yellow label warning "DON'T PUT YOUR FOOT ON – DON'T PUT OBJECTS ON" is located next to moving parts: it warns against putting any object or limb in the chair movement area in order to avoid crushing damages.

## 2. TRANSPORT AND INSTALLATION

### 2.1. Handling and removal from the packaging

The chaise can be raised and moved manually by two or four people, taking it exclusively from the 4 support feet.

The weight of the chair is equal to 68 kg.

The chair is shipped packed in a carton box. Open the box so you can freely raise the chair; remove it from the packaging, taking it from the base and pull it upwards.

#### ATTENTION!

- Handling and moving operations have to be done by qualified and instructed personnel.
- It is strictly forbidden to remove the chair by lifting it from the top (mattress).
- Do not use knives or box cutters to open the box so as not to risk damaging the chair and in particular the pudding.
- Before using the chair, clean it entirely according to the procedure shown at § 9.4. Cleaning

### 2.2. Flooring

The floor on which the chair will be placed must be solid, smooth, horizontal and adequate to support its weight and ensure the stability.

### 2.3. Lighting

The user must ensure that the lighting of the work areas is sufficient to allow a good view of the operations and all elements of the chair; the level of lighting should be in accordance with the UNI EN 12464-1; in particular should be avoided shadows and annoying glare.

### 2.4. Positioning

The position of the chair compared to fixed obstacles must be such as to allow the easy passage for the use and maintenance; in particular, the minimum space to allow the passage of persons shall be equal to 650 mm (in accordance with UNI EN 349:2008). The minimum distance between the completely lowered back and fixed obstacle has to be 500 mm, in order to avoid the body to be crashed by the chair.

### 2.5. Levelling

The chair is provided with 4 adjustable feet that enable to level it perfectly and to compensate any failing of the flooring. The regulation can be done by screwing or unscrewing the feet.

### 2.6. Power supply

The system of power supply must have voltage and frequency equal to the rated voltage and frequency of the power supply of the device and be appropriate to the installed capacity, as indicated on the identification plate of the chair.

Voltage	100 V / 240 V
Frequency	50/60 Hz
Power / Absorption	4 Amax (1,7 A)
Power supply	Alternate Current (AC) One Phase L+N
Duty cycle	2 min work / 18 min at 10% maximum
IP Protection	IP54

Control box class II

## The power supply must be supplied to the unit by a line:

- Protected against overcurrent by an automatic circuit breaker (circuit breaker) or fuses, properly sized; the operating current of the protections must be as low as possible taking into consideration the maximum absorption of the device;
- Equipped with a device sensitive to insulation failure (differential) with operating current low enough; this device must be positioned downstream of the overcurrent protective device (or be integrated with it);
- Having cables with section suitable for the power absorbed by the appliance.

To avoid the risk of electric shock, this appliance **must only be connected to a power supply with protective earth (grounding).**

## 2.7. Connections

**START (ON):** plug the power cable into a suitable electric socket. If there is incompatibility between the outlet and the plug of the chair, ask qualified personnel to replace it with a suitable one.

**STOP (OFF):** Remove the plug from the electric socket.

**ATTENTION!** Pull out by grabbing THE PLUG, NOT THE CORD

**WARNING!** It is strongly recommended to insert the plug into a single outlet exclusively reserved to the device. We do not recommend the use of multiple sockets (power strip), adapters and patch cords. The cable must not be bent.

**WARNING!** Before plugging in the cable, make sure that it is not twisted or bended and make sure that the protective sheath is undamaged.

**WARNING!** The chair must be positioned in order to ease the removal of the electric plug.

**WARNING!** To avoid the risk of electrical shock, this chair must exclusively be plugged in power supply grids with protective ground.

## 3. ENVIRONMENTAL WORKING CONDITIONS AND STORAGE

The chair has been designed and built to operate in a closed and dry room. The ambient operating condition are:

- temperature: 5° ÷ 40° C 
- humidity: 30% ÷ 75% without condensation. 

If the environmental conditions are particularly critical it is advisable to provide the room with an adequate air conditioning system to bring the humidity and temperature values within acceptable limits.

**WARNING!** Do not expose the chair to an excessively humid environment. If the chair gets wet, remove the plug from the outlet and have it checked by an Authorized Service Centre.

**WARNING!** The chair was not conceived to be used in settings with explosive atmospheres, in presence of thin powders or corrosive gases, acids, corrosive agents or salt.

**WARNING!** It is recommended to check the adequacy of the electromagnetic performance of the devices adjacent to the chair, observing that it continues to operate normally.

- Not intended to be used in oxygen rich environments.

- Not intended to be used with flammable anesthetics or in conjunction with flammable agents.

**STORAGE!** For the correct functioning of the electrical equipment the storage of the chair must be at a temperature between -25° C and +55° C, and only for short periods not exceeding 24 hours, to +70° C. The chair has no special requirements for storage, provided that it takes place in a closed and dry rooms; this condition should also be ensured during all phases of transport.

## 4. PRELIMINARY INSPECTIONS

Before using the chair, please check that:

- the equipment does not present any damage arising from transport or handling; if needed, please contact the manufacturer Brusaferrri & C to solve such inconveniences;
- all packing (carton box, foam, plastic) have been removed;
- the voltage of your cabin is the same as the voltage showed in section 2.6 Power Supply;
- all control devices (electrical or manual) work correctly

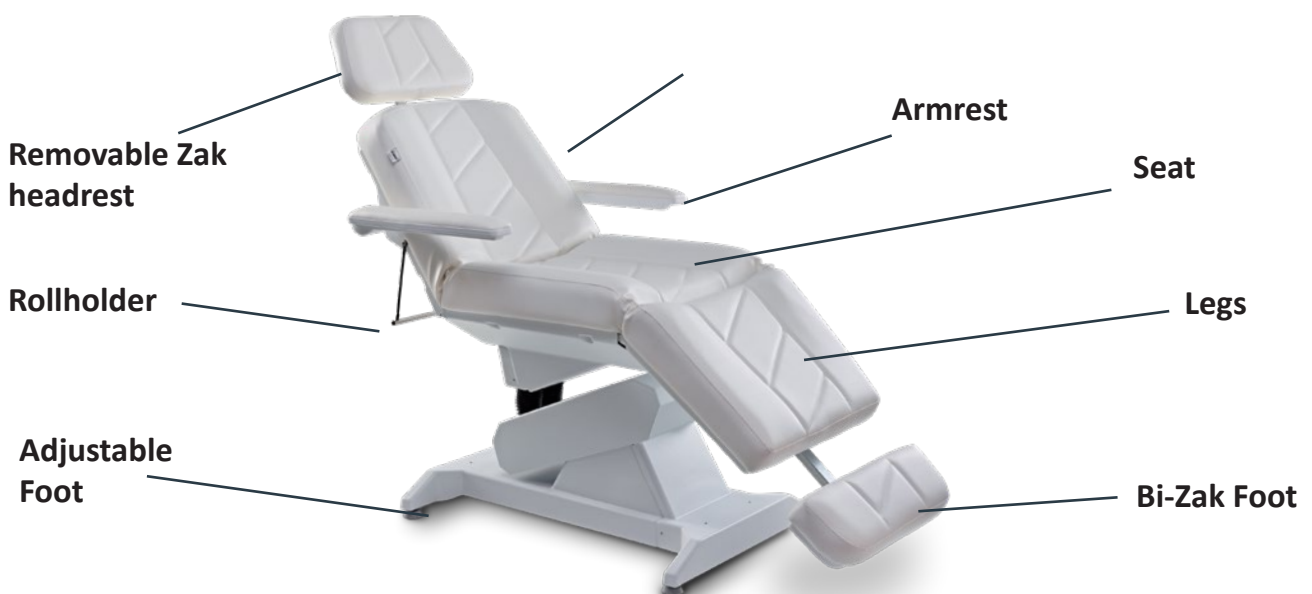
- For the installation, please contact qualified personnel only.
- Before the setting at work of the chair itself, run a few idling cycles test, in conditions of safety, by qualified personnel.

## 5. DESCRIPTION OF THE CHAIR

### LEMI 4

### LEMI 4 BI-ZAK /LEMI SYNCRO BI-ZAK


## LEMI MED

## 6. OPTION ON DEMAND

### 6.1. Head rest for Hair Transplantation

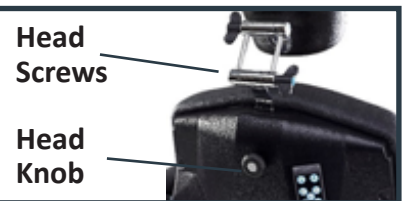To adjust / remove the Anto head rest, suitable for hair transplantation, turn the head knob and adjust by the 2 head screws.


### 6.2. Face hole headrest

To adjust / remove face hole headrest, turn the head knob and adjust by the head handle. Velcro® fixes the upholstered cushion to the wooden support.


### 6.3. Wireless pedal (without cable)

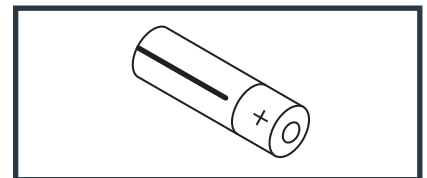If the chair was equipped with a wireless pedal, follow the instruction in § 7.3. Controls for the installation and proper use. (Max distance 3mt)  
The wireless pedal works with two AAA batteries: unscrew the two screws on the plate that is on the top of the pedal to replace the batteries.

### 6.4. Flat Armrest

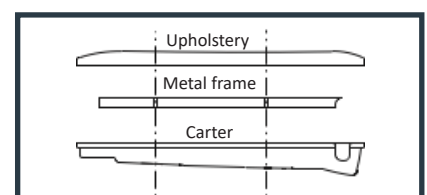The flat armrests are assembled with screws supplied with the carter and the mattress. Position the padding (upholstery) on the metal frame; then position the abs carter under the metal frame and fix it with two screws 5x35. Pay attention to combine the carter and the padding.

**WARNING!** These armrest were only meant to support the arm. **It is strictly forbidden to seat on the armrests.** Any damage caused by improper use, would not be considered under warranty.


### 6.5. Battery

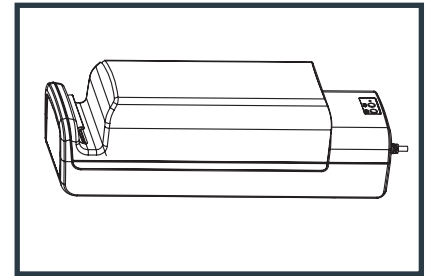The battery (5.0Ah capacity) is dual-function:

**PROMISCUOUS:** the battery acts as an auxiliary in the event of a power failure

**ONLY WITH BATTERY:** the battery works without connecting the chair to the power cable.

The battery user manual is provided separately when the accessory is selected.

### 6.6. LEMI 4 Bi-Zak version

The LEMI 4 BI-ZAK version is equipped with an adjustable and removable headrest and footrest.

**HEADREST:** To adjust the inclination, push or pull; while to extend or remove, unlock the knob located at the rear of the backrest.

**FOOTREST:** To extend or remove the foot, unlock the knob located under the legs.

### 6.7. LEMI MED version

The standard version of LEMI MED is equipped with an adjustable and removable headrest and footrest, two guides mounted on the seat and two on the back for fixing any medical accessories and a roll holder.

### 6.8. Rollholder

The chair can be equipped with a rollholder, which is installed by using specific strings fixed to two screws on the backrest.

### 6.9. Medical accessories

Accessory	Position of the guides	Clamps to use to fix the accessory
Thigh Support	On the seat	Fixed
Heel Support	On the seat	Fixed
Feet Support	On the seat	Fixed
Medical Armrests	On the back	Rotatable
Blood taking armrests	On the seat	Fixed
Curtain holder	On the back	Rotatable
I.V. Rod	On the back	Rotatable
Inox Metal Tray	On the seat	Fixed

You must insert the accessories in the slots of the clamps and fix to the guides by screwing and unscrewing the handle. As for the blood-sample taking armrests, the thigh/heel/feet supports and the metal tray, once they are properly installed it is possible to turn the handles without loosening the clamps; simply pull the handles and place them so that they do not bother while using the chair.

## 6.10. Heating

The heating system of the mattress can be regulated by the "HEAT" button panel, which is on the backrest.

**SWITCH ON:** Button 1 (red led ON): activation of the LOWEST temperature (40 ° C)

**Button 1 + Button 2:** activation of the HIGHEST temperature (47 ° C)

**SWITCH OFF:** Button 1 + Button 2 (red led OFF)

**WARNING!** Do not place ANY object on the seat and / or back when the heating is on to avoid excessive overheating of the affected area.

**WARNING!** Do not cover the product with more than one mattress cover, make sure you use quality products.

**WARNING!** ALWAYS TURN OFF THE HEATING WHEN THE CHAIR IS NOT IN USE.

## 6.11. Wheels

The 4 wheels with brake (∅ 50mm) are installed in place of the feet.

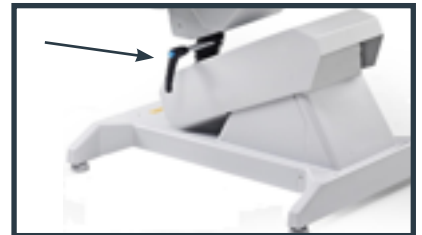
**WARNING!** ALWAYS block the brake of each wheel before the client sit on the chair.

## 6.12. Chair Rotation

The upper part of the chair can rotate to 180° (90° left + 90° right). To rotate the chair, turn the handle situated under the basement, fix the chair into the required position and block the rotation by tightening the handle clockwise.

# 7. INSTRUCTION FOR USE

## 7.1. Operator

The chair is controlled by an operator through the foot pedal (or the hand control); the operator must position themselves so as to have complete view of the controlled parts and of the patient.

**WARNING!** During the functioning of the chair the operator should always remain in the vicinity of the controls; it is forbidden to leave the chair unattended during its operation.

Operators should pay attention to avoid creating potentially dangerous situations for their own or other's safety.

All people working on the chair must be properly trained on the correct methods of work and informed about its proper functioning. They also need to be aware of the parts of this documentation relevant to their work.

Operators must not wear clothes with drapes that could get caught by moving part of the chair. Proper clothing is made of close-fitting garments, especially on the upper limbs. Nor the operator and the patient must wear rings, necklaces, bracelets, wristwatches, ties or other objects that could get caught by moving parts of the machinery.

In case of allergy, cover the chair with a sheet.  
The operator must stand in order to always control all the adjusted parts and also the patient.

## 7.2. Use of mattress

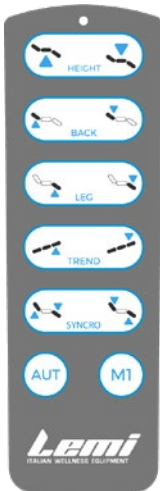
IMPORTANT! It is mandatory to cover and protect the mattress with disposable paper covers or washable sponge covers (better if sponge + PVC) for its correct and hygienic preservation. The upholstery could be damaged by the normal heat of the skin or by a prolonged use of essential oils or cosmetic product. Any replacement caused by damages due to misuse won't be considered under warranty.


## 7.3. Controls

The chair can be controlled by a pedal with cable, a wireless pedal or a hand control. All the controls work only when activated: the chair movement will stop as soon as the operator releases the button/pedal, except for the AUT position which can be stopped by pressing any button/pedal. Max 6 wireless controls can be available for one single bed.

The operator only can use controls (foot and hand control) to move the examination chair

<p>A</p> <p>B</p> <p>C</p> <p>D</p> <p>E</p> <p>AUT</p>		<p>A1</p> <p>B1</p> <p>C1</p> <p>D1</p> <p>E1</p> <p>M1</p>	<h3>HANDSET CONTROL (LEMI 4/LEMI 4 BI-ZAK/LEMI MED)</h3> <p><b>Button A:</b> Chair Up <b>Button A1:</b> Chair Down <b>Button B:</b> Back Up <b>Button B1:</b> Back Down <b>Button C:</b> Legs Up <b>Button C1:</b> Legs Down <b>Button D:</b> Trendelenburg Up <b>Button D1:</b> Trendelenburg Down <b>Button E:</b> Synchronized Back Up and Legs Down</p>	<p><b>Button E1:</b> Synchronized Back Down and Legs Up <b>Button AUT:</b> automatic return to original position of the chair, to allow customer to get up or down easily. <b>Button M1:</b> Memory position 1</p>
---	--	---	---	--

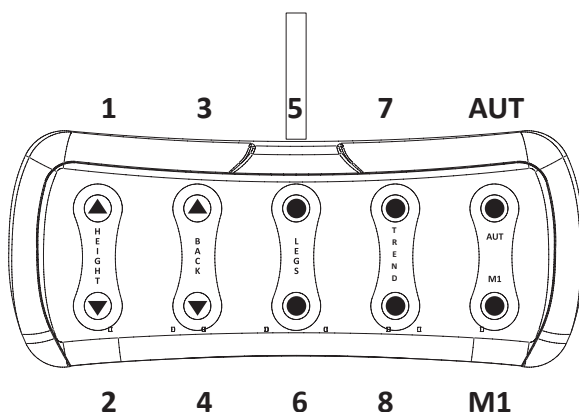
### FOOT PEDAL CONTROL /WIRELESS (LEMI 4/LEMI 4 BI-ZAK/LEMI MED)

**Pedal 1:** Chair Up  
**Pedal 2:** Chair Down  
**Pedal 3:** Back Up  
**Pedal 4:** Back Down  
**Pedal 5:** Legs Up  
**Pedal 6:** Legs Down  
**Pedal 7:** Trendelenburg Up  
**Pedal 8:** Trendelenburg Down  
**Pedal AUT:** automatic return to original position of the chair, to allow customer to get up or down easily.  
**Pedal M1:** Memory position 1

Once the treatment is ended, before the patient/client gets up, keep the chair to the initial position by pressing the AUT button/pedal.

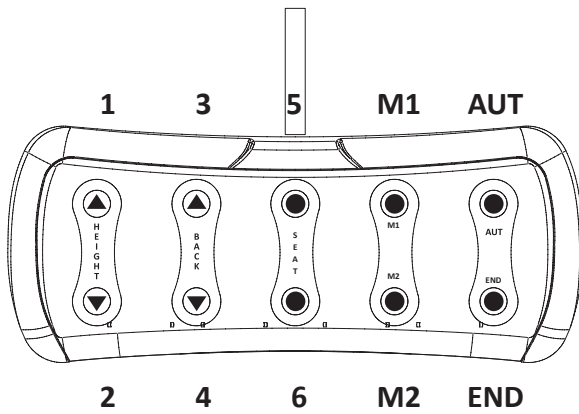
## FOOT PEDAL CONTROL /WIRELESS (LEMI SYNCRO BI-ZAK)

- Pedal 1:** Chair Up
- Pedal 2:** Chair Down
- Pedal 3:** (synchronized) backrest-legrest
- Pedal 4:** (synchronized) backrest-legrest (return)
- Pedal 5:** Trendelenburg/tilting Up
- Pedal 6:** Trendelenburg/tilting Down
- Pedal M1:** Memory position 1
- Pedal M2:** Memory position 2
- Pedal AUT:** automatic return to original position of the chair, to allow customer to get up or down easily.
- Pedal END:** automatic return from any position to the flat position (flat and minimum height)

Once the treatment is ended, before the patient/client gets up, keep the chair to the initial position by pressing the AUT button/pedal.


## HANDSET CONTROL (LEMI SYNCRO BI-ZAK)

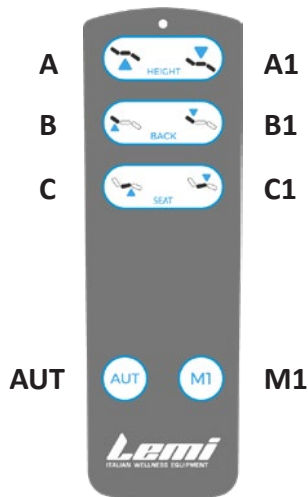- Button A:** Chair Up
- Button A1:** Chair Down
- Button B:** (synchronized) backrest-legrest
- Button B1:** (synchronized) backrest-legrest (return)
- Button C:** Trendelenburg/tilting Up
- Button C1:** Trendelenburg/tilting Down
- Button AUT:** automatic return to original position of the chair, to allow customer to get up or down easily.
- Button M1:** Memory position 1

### 7.4. Leg Extension (LEMI 4 BI-ZAK and LEMI MED)

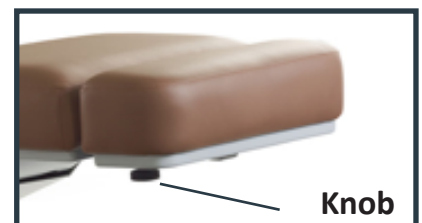To stretch out the legrest, turn the knob situated near the leg and pull the upholstery to reach the required position; then tighten the knob.

### 7.5. Head adjustment (LEMI 4 BI-ZAK/LEMI SYNCRO BI-ZAK/LEMI MED)

To adjust / remove the head, turn the head knob

### 7.6. Head adjustment (LEMI 4)

To adjust the head inclination, press the button under the back rest (push)

### 7.7. Legs adjustment (LEMI SYNCRO BI-ZAK)

To adjust the legs separately from the backrest: support the final part of the upholstery leg with your hand, adjust it at the correct position and fix it by closing the LEGS HANDLE you can find on the side of the legs. **ATTENTION: the legs handle must only turn clockwise/anticlockwise. DO NOT PRESS THE BLUE BUTTON.**

### 7.8. Foot adjustment (LEMI 4 BI-ZAK/ LEMI MED)

To adjust / remove the foot, turn the foot knob

## 8. USE OF THE CHAIR

 = 200 kg (max. patient weight)

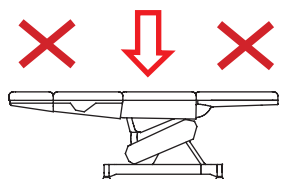
 = 2500 N (safety load)

**ATTENTION!** Only one person can get up on the chair; the person must go up/down when the chair is in the basic position going up on the seat.

**WARNING!** Get on the chair always by sitting properly in the central part of the seat. Do not sit or lean on the side of the feet or of the head.

**WARNING!** The limbs of the patient must always be on the chair. The operator must always inform the patient when he moves the chair.

**WARNING!** Be careful to make sure that the chair or its electrical parts are not wet and that liquids become not stagnant on the chair itself; immediately wipe up any liquids – including physiological liquids of the patient – which may end up on the chair.

**WARNING!** If a patient is let alone for a short time, it is necessary to place the chair in the lowest position with respect to the walking floor, in order to reduce risk of damage in case of fall

To avoid shock hazard, do not connect or disconnect the power plug with wet hands. Do not use the chair barefoot.

## 9. MAINTENANCE

The chair has been designed and constructed so as to reduce as much as possible the necessary maintenance and ensure proper functioning over the years without any problems. During any intervention on the chair, it is recommended to follow the instructions given in this section.

**WARNING! All interventions on the chair must be carried out by competent and trained personnel on the methods to operate correctly and safety. Do not make any action on the chair moving, and always check that the plug connecting to power supply is disconnected; bring the power plug next to you during the execution of the intervention on the chair so you can check at any time the actual separation.**

### 9.1. General Checks

Constantly check the general condition of the chair: the integrity of the protections, the electrical equipment and the condition of the pads.

**WARNING! Do not use the chair if any component is found to be not perfectly intact or in service. In this case, put the chair out of service by disconnecting the power plug immediately and contact an Authorized Service Centre.**

**WARNING! Do not modify this chair without authorization of the manufacturer. If this chair is modified, appropriate inspection and testing must be conducted to ensure its continued safe use.**

### 9.2. Extraordinary Maintenance

Any extraordinary maintenance must be performed by qualified personnel of the technical service Brusaferrri & C. or Service Centre authorized by the manufacturer.

Information about the Authorized Service Centre to which you can turn, can be obtained directly from the manufacturer Brusaferrri & C srl – phone 0374/363068-69 – mail: [service@lemigroup.it](mailto:service@lemigroup.it)

For a correct and quickly resolution of the problem, it is necessary to communicate the model and the serial number of the chair and the description of the defect, or what are the visible problems, and under what condition it occurs.

The broken chair, if under warranty, must be sent to the Service Centre together with the certificate of guarantee, transport charges must be paid by the purchaser.

**WARNING! Any form of warranty expires if interventions are carried out on the chair by someone not authorized by Brusaferrri & C.**

### 9.3. Authorized Personnel

Repairs and maintenance of the chair can be carried out exclusively by competent personnel, that is of Brusaferrri & C. or authorized by it. Information on the staff to contact for possible interventions can be obtained directly from Brusaferrri & C.

### 9.4. Cleaning

Each time you make a treatment, always cover the mattress with towels or disposable towels before the patient sit on the chair (See §7.2. Use of the mattress).

Pay particular attention to the jeans that can stain the covering without any possibility of being removed.

Clean the chair before each use, paying particular attention to remove all traces of physiological fluids of the previous patient. To clean the chair use a cloth and lukewarm water (if necessary diluted neutral detergent). If necessary, use a non-aggressive disinfectant, then wipe off with a clean dry cloth.

Do not use products such as alcohol, acetone, bleach, paint thinner, gasoline.

#### WARNING!

- Be careful not to spray liquids on the electrical equipments, pedal/push-button strip and on the protections of the chair.
- You must follow the handling directions given in the safety data sheets of the detergents used.
- While during the cleaning operation of the chair, inhaling powders are produced, use a proper protective device and air adequately the room.
- Use proper individual protection devices

## 10. IN CASE OF AN ACCIDENT

Any serious accident occurring in relation to this device must be immediately reported to the manufacturer and the competent authority of the member state in which the user and / or patient is established.

Do not remove protections in any case; do not use the chair if the protections are damaged or not correctly fixed.

Do not put limbs near to the moving parts of the chair.

## 11. SPARE PARTS

All the spare parts must be requested to Brusaferrri & C. that will supply them directly or will give you indications on the place where they can be found. Please note that any extraordinary maintenance must be performed by personnel of Brusaferrri & C. or authorized by it.

**WARNING! Any form of warranty expires if they are used spare parts other than those originally fitted.**

## 12. DISMANTLE

If the chair becomes inoperative or is not to be used for an extended period of time, it is recommended to remove it from service and remove the plug from the socket. All materials used to manufacture the chair are not toxic or hazardous to the health of operators, so they can be handled without special precautions. In case of disable of the chair you should perform a separation of different materials for subsequent reuse or differentiated elimination. All electrical and electronic products should be separately disposed, using specific collection facilities appointed by the Government or the Local Authorities.

## 13. DISPOSAL OF OBSOLETE EQUIPMENT

The correct disposal of the obsolete equipment will help to prevent potential negative effects on human health and the environment. For information, contact your Local Council or the Waste Disposal Service.

- **The dismantling of the chair at the end of its functional life must be done by qualified personnel trained about the correct way of handling – and if necessary of dismantling the chair.**
- **Use only suitable equipment and lift appliances in accordance with the National Regulations and Legislative Law in force.**
- **Having to dismantle the chair or to move it in another place, please contact the manufacturer Brusaferrri & C – Italy.**

## IMPORTANT NOTICE

# HOW TO CLEAN THE ABS PLASTIC COVERING

It is forbidden to use the following products which could seriously damage the plastic covering, provoking cracks and the subsequent breaking:

- Degreasing solvent-based agents: e.g. ACETONE, DENATURED ETHYL ALCOHOL, TRICHLOROETHYLENE, FUEL, TURPENTINE...
- Watered-down acids (HYDROCLORIDRIC ACID, NITRIC ACID) or products containing watered-down acids
- Watered-down bases (CAUSTIC SODA, SODIUM HYPOCHLORITE, AMMONIA, BLEACH) or products containing weak bases
- Fluorinated hydrocarbons (e.g. LEAKS FROM REFRIGERANT GAS)
- Food-grade oils (e.g. OLIVE OIL, SEEDS OIL)
- Lubricating mineral-based oils

To CLEAN the coverings we recommend:

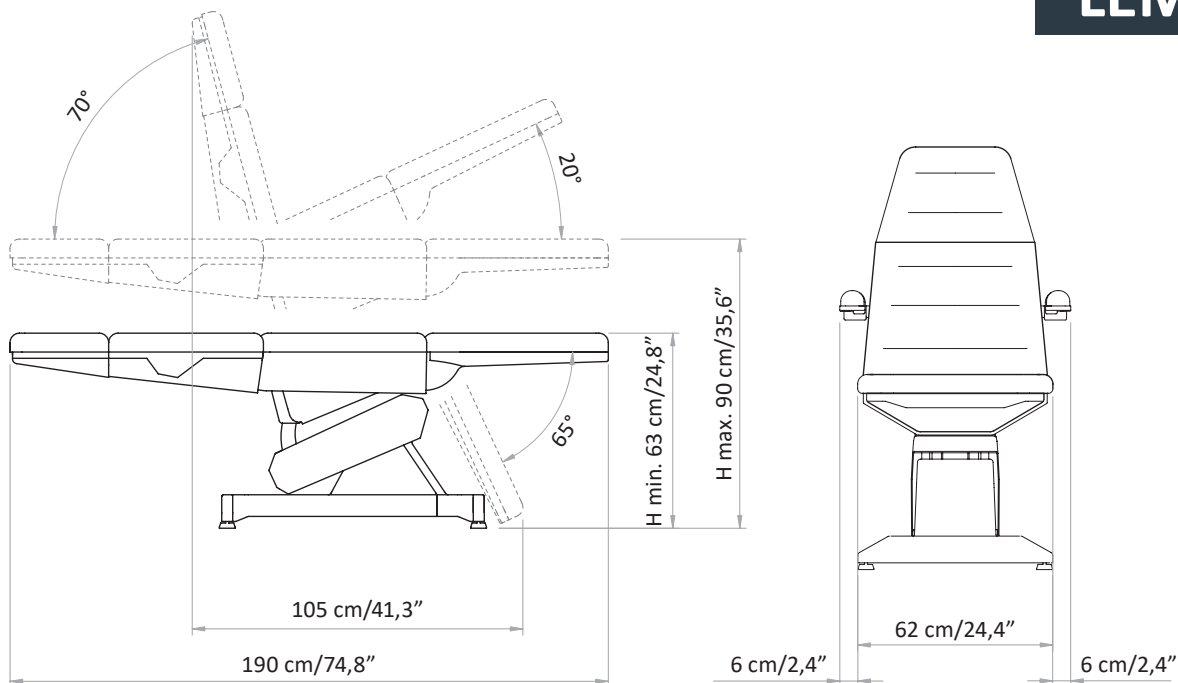
- Soap water
- Water and vinegar
- Non abrasive cream cleanser

Always rinse with clean water and dry with a soft, not abrasive cloth.

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---

This publication or parts thereof may not be reproduced, stored in a storage machine, transmitted, transcribed, or translated into any language, usual or computer, in any form or by any means, electronic, mechanical, magnetic, optical, chemical, manual or otherwise, without the express written permission of the manufacturer, Brusaferrri & C. The manufacturer shall provide warranty with respect to the contents of this publications, with the express prohibition of the use of the same publication for any other purpose whatsoever, commercial, practical, technical-scientific, unrelated to the functioning of the equipment to which it relates for which this issue has been written. The warranty on the accuracy of the information contained herein is provided on condition that they are strictly observed by the user of the device, all the prescriptions contained in this documentation. Moreover Brusaferrri & C. reserved the right to revise this publication and to make changes in the content without obligation to notify any person or organization.

These revised publications are available on request by Brusaferrri & C.

©2023 Brusaferrri & C. – All rights reserved.

Designed and manufactured by:

***Lemi***<sup>®</sup>  
ITALIAN WELLNESS EQUIPMENT

Via Mara Maretti Soldi, 13  
26011 Casalbuttano (CREMONA) ITALY  
Phone +39-0374-363069 Fax +39-0374-363071  
info@lemigroup.it service@lemigroup.it  
www.lemigroup.it