PODO MIX LINE - PODO 5

ORIGINAL INSTRUCTIONS - REV. 1 / 2023

PODO MIX 1-2 M

PODO MIX 3 M

PODO 5

INSTRUCTION FOR USE pag. 3

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EN

Congratulation for having chosen an equipment LEMI produced by Brusaferrri & C.

This chair has been designed and built using state of the art technologies and processes to guarantee the best reliability in the time and ensure maximum safety.

For a correct and safe operation of the chair is essential to carefully read and observe what is indicated in this manual. All documentation accompanying the chair – and in particular this manual – must be carefully preserved for future reference.

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Symbols meaning

Non-compliance of this procedure could cause damages to the chair or its parts.

Non-compliance of this procedure could cause health damages or injury to the operator or people near the chair

1. INTRODUCTION

The successful and safe operation of the chair is guaranteed only if it is used in accordance with the specifications in this manual and, in general, in the accompanying documentation; therefore it is essential to carefully read and keep all the relative documentation. Although the chair is equipped with active and passive safety features, not all risks can be avoided due to its incorrect use.

You must always make sure that all users have fully understood the rules of use; the manufacturer declines any responsibility for damage to persons or property resulting from improper use of the device.

Do not remove or damage the labels, notices and warnings placed on side of the chair. In case you need to restore them, please contact the manufacturer. The manufacturer Brusaferrri & C., declines any and every responsibility for failure to observe the rules of safety and prevention described in the various sections of this manual and for any damage caused by improper use. All works on the chair (maintenance, adjustments, repairs, cleaning) must be carried out by suitably trained personnel and in accordance with what is indicated in this manual.

1.1. Destinazione d'uso

This chair is suitable for professional footcare treatment. **The estimated lifetime for this chair is 10 years.**

WARNING! It is forbidden any use of the chair other than that described in this manual. Brusaferrri & C. declines any responsibility for malfunctions or damages to persons or property due to uses other than those specified in this manual.

The chair should be used only by personnel who are informed about the use mode, operation and its safety equipment.

1.2. Preservation of the manual

This operating manual should be kept for future reference. We recommend:

Keep the manual in a place accessible and known to all operators, protected from moisture and heat and protected from the direct sun beams. Use the manual in order not to damage any part of the content: do not remove, tear or modify any part of the manual.

In the case of sale or transfer of the chair to another person, this manual and its annexes should be delivered intact to the new users.

WARNING! Please read this operation manual before using the chair. Anyone using the chair should be properly informed on the parts of this manual relevant to the operations to be performed.

1.3. Labelling of the chair

The chair is provided with labels with the following data, in conformity with actual rules (see § 14. Label Legend)

WARNING! It is recommended not to remove the labels on the machine; labels must be preserved well fixed, whole and easy to read.

WARNING! The data present on each label, must be reported in case of need of assistance or spare parts.

1.4. Warning messages

The chair is provided with signs showing phrases or symbols; pay attention to the warnings on the equipment.

The symbol that warns against the risk of electrocution is located next to the plug of the electric supply.

The yellow label warning "DON'T PUT YOUR FOOT ON – DON'T PUT OBJECTS ON" is located next to moving parts: it warns against putting any object or limb in the chair movement area in order to avoid crushing damages.

2. TRANSPORT AND INSTALLATION

2.1. Handling and removal from the packaging

The chair can be raised and moved manually by two or four people, taking it exclusively from the 4 support feet.

The weight of the chair is equal to 77 kg.

The chair is shipped packed in a carton box. Open the box so you can freely raise the chair; remove it from the packaging, taking it from the 4 feet and pull it upwards.

ATTENTION!

- Handling and moving operations have to be done by qualified and instructed personnel.
- It is strictly forbidden to remove the chair by lifting it from the top (mattress).
- Do not use knives or box cutters to open the box so as not to risk damaging the chair and in particular the pudding.
- Before using the chair, clean it entirely according to the procedure shown at § 9.4. Cleaning

2.2. Flooring

The floor on which the chair will be placed must be solid, smooth, horizontal and adequate to support its weight and ensure the stability.

2.3. Lighting

The user must ensure that the lighting of the work areas is sufficient to allow a good view of the operations and all elements of the chair; the level of lighting should be in accordance with the UNI EN 12464-1; in particular should be avoided shadows and annoying glare.

2.4. Positioning

The position of the chair compared to fixed obstacles must be such as to allow the easy passage for the use and maintenance; in particular, the minimum space to allow the passage of persons shall be equal to 650 mm (in accordance with UNI EN 349:2008). The minimum distance between the completely lowered back and fixed obstacle has to be 500 mm, in order to avoid the body to be crashed by the chair.

2.5. Levelling

The chair is provided with 4 adjustable feet that enable to level it perfectly and to compensate any failing of the flooring. The regulation can be done by screwing or unscrewing the feet.

2.6. Power supply

The system of power supply must have voltage and frequency equal to the rated voltage and frequency of the power supply of the device and be appropriate to the installed capacity, as indicated on the identification plate of the chair.

Voltage	100 V / 240 V (Podo Mix 3M/1-2M) 110 V / 230 V (Podo 5)
Frequency	50/60 Hz
Power / Absorption	4 Amax (1,7 A) (Podo Mix 3M/2M) 1,5 Amax (0,6 A) (Podo Mix 1M) 0,7 - 1 Amax (Podo 5)
Power supply	Alternate Current (AC) One Phase L+N
Duty cycle	2 min / 18 min at 10% maximum
IP Protection	IP54

Control box class II

The power supply must be supplied to the unit by a line:

- Protected against overcurrent by an automatic circuit breaker (circuit breaker) or fuses, properly sized; the operating current of the protections must be as low as possible taking into consideration the maximum absorption of the device;
- Equipped with a device sensitive to insulation failure (differential) with operating current low enough; this device must be positioned downstream of the overcurrent protective device (or be integrated with it);
- Having cables with section suichair for the power absorbed by the appliance.

To avoid the risk of electric shock, this appliance **must only be connected to a power supply with protective earth (grounding).**

2.7. Connections

START (ON): plug the power cable into a suichair electric socket. If there is incompatibility between the outlet and the plug of the chair, ask qualified personnel to replace it with a suichair one.

STOP (OFF): Remove the plug from the electric socket.

ATTENTION! Pull out by grabbing THE PLUG, NOT THE CORD

WARNING! It is strongly recommended to insert the plug into a single outlet exclusively reserved to the device. We do not recommend the use of multiple sockets (power strip), adopters and patch cords. The cable must not be bent.

WARNING! Before plugging in the cable, make sure that it is not twisted or bended and make sure that the protective sheath is undamaged.

WARNING! The chair must be positioned in order to ease the removal of the electric plug.

WARNING! To avoid the risk of electrical shock, this chair must exclusively be plugged in power supply grids with protective ground.

3. ENVIRONMENTAL WORKING CONDITIONS AND STORAGE

The chair has been designed and built to operate in a closed and dry room. The ambient operating condition are:

- temperature: 5° ÷ 40° C 
- humidity: 30% ÷ 70% without condensation. 

If the environmental conditions are particularly critical it is advisable to provide the room with an adequate air conditioning system to bring the humidity and temperature values within accepchair limits.

WARNING! Do not expose the chair to an excessively humid environment. If the chair gets wet, remove the plug from the outlet and have it checked by an Authorized Service Centre.

WARNING! The chair was not conceived to be used in settings with explosive atmospheres, in presence of thin powders or corrosive gases, acids, corrosive agents or salt.

ATTENTION! It is recommended to check the adequacy of the electromagnetic performance of the devices adjacent to the chair, observing that it continues to operate normally.

- Not intended to be used in oxygen rich environments.
- Not intended to be used with flammable anesthetics or in conjunction with flammable agents.

STORAGE! For the correct functioning of the electrical equipment the storage of the chair must be at a temperature between -25° C and +55° C, and only for short periods not exceeding 24 hours, to +70°C. The chair has no special requirements for storage, provided that it takes place in a closed and dry rooms; this condition should also be ensured during all phases of transport.

4. PRELIMINARY CHECK

Before using the chair, please check that:


- the equipment does not present any damage arising from transport or handling; if needed, please contact the manufacturer Brusaferrri & C to solve such inconveniences;
- all packing (carton box, foam, plastic) have been removed;
- the voltage of your cabin is the same as the voltage showed in section 2.6 Power Supply;
- all control devices (electrical or manual) work correctly


- For the installation, please contact qualified personnel only.
- Before the setting at work of the chair itself, run a few idling cycles test, in conditions of safety, by qualified personnel.

5. DESCRIPTION OF THE CHAIR

PODO MIX 1-2M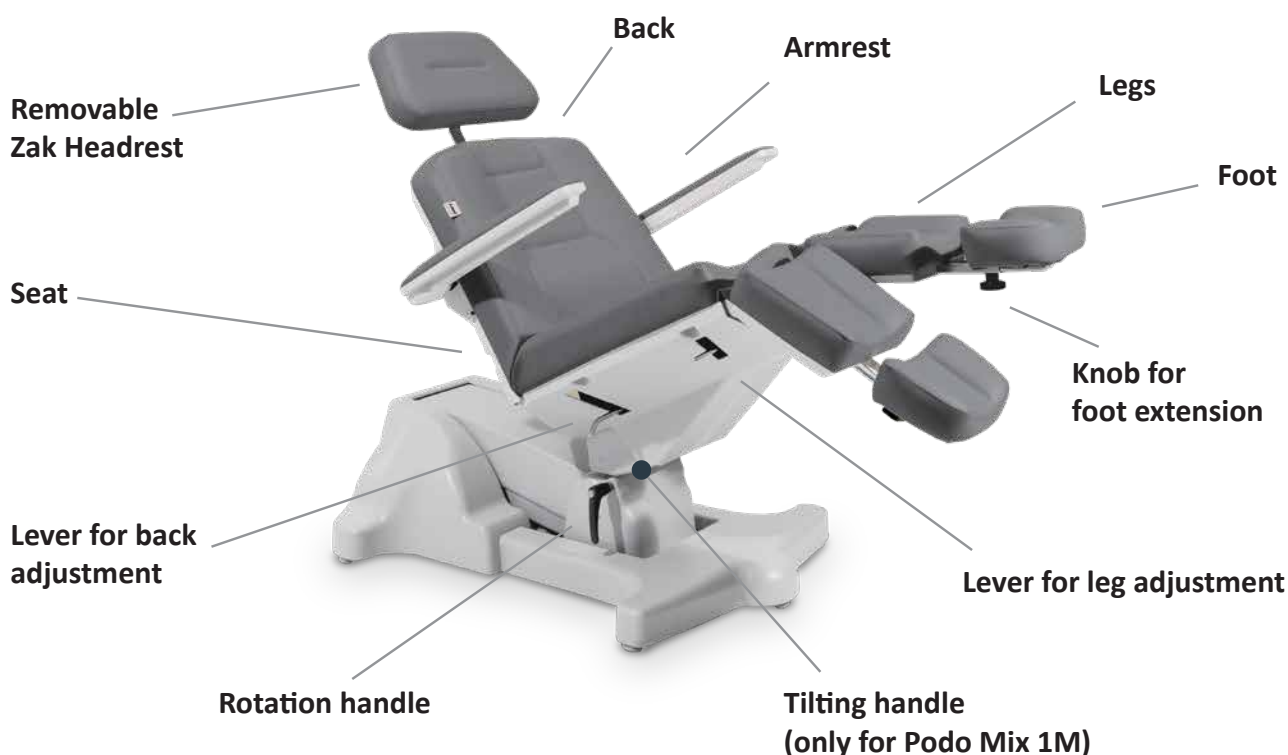

PODO MIX 3M

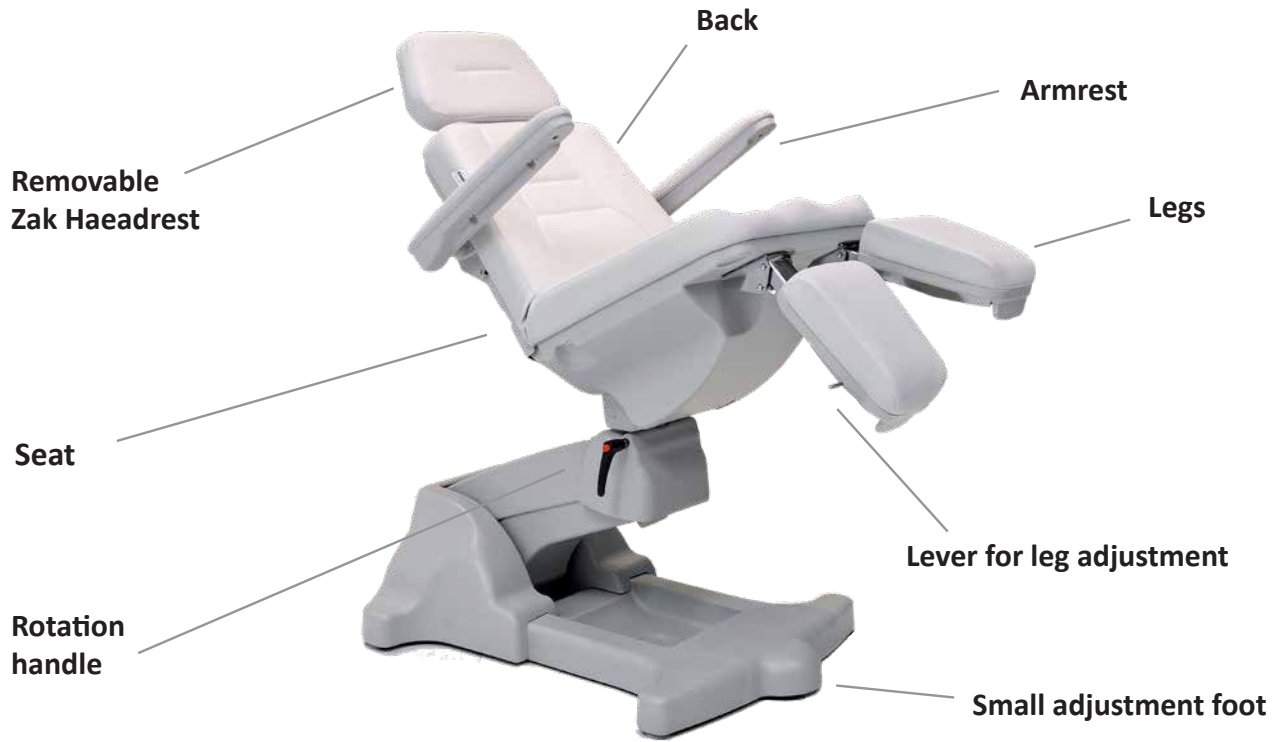
PODO 5

6. OPTION ON DEMAND

6.1 Wireless pedal (without cable) - Podo Mix 3M only

If the chair is equipped with a wireless pedal, follow the instruction in § 7.3. Controls for the installation and proper use. (Max distance 3mt)
The wireless pedal works with two AAA batteries: unscrew the two screws on the plate that is on the top of the pedal to replace the batteries.

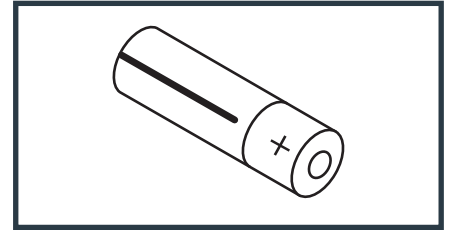6.2. Wheels

The 4 wheels with brake (Ø 50mm) are installed in place of the feet.

WARNING! ALWAYS block the brake of each wheel before the client sits on the chair.

6.3. Rollholder

The chair can be equipped with a rollholder, which is installed by using specific strings fixed to two screws on the backrest.

6.4. Foot tray

Insert the white foot tray on its support fixed on the leg.
Pay attention to remove foot tray when you move down the legs in order to avoid any crushing.

7. INSTRUCTIONS FOR USE

7.1. Operator

The chair is controlled by an operator through the foot pedal control with cable (standard) or wireless (option only for Podo Mix 3M); the operator must position themselves so as to have complete view of the controlled parts and of the patient.

WARNING! During the functioning of the chair the operator should always remain in the vicinity of the controls; it is forbidden to leave the chair unattended during its operation.

Operators should pay attention to avoid creating potentially dangerous situations for their own or other's safety.

All people working on the chair must be properly trained on the correct methods of work and informed about its proper functioning. They also need to be aware of the parts of this documentation relevant to their work.

Operators must not wear clothes with drapes that could get caught by moving part of the chair. Proper clothing is made of close-fitting garments, especially on the upper limbs.

Nor the operator and the patient must wear rings, necklaces, bracelets, wristwatches, ties or other objects that could get caught by moving parts of the machinery.

In case of allergy, cover the chair with a sheet.
The operator must stand in order to always control all the adjusted parts and also the patient.

7.2. Use of the mattress


IMPORTANT! For a correct conservation of the mattress (in particular the covering) and for hygienic reasons it is essential to cover and protect the mattress with disposable paper / cartene covers and washable sponge mattress covers (better if sponge + PVC). In fact, the upholstery could be damaged with the natural heat of the skin and with the prolonged use of products containing oils or alcohol. Any replacements for damage caused by improper use will not be considered under warranty.

7.3. Controls

The chair can be controlled by a foot pedal control. All controls are maintained action, that is releasing the control pedal the motion stops.

The operator only can use controls (foot control) to move the chair.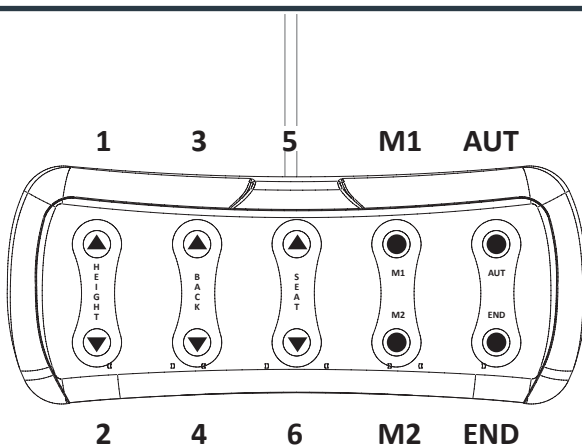

PODO MIX 3M PEDAL CONTROL

Pedal 1: Height UP

Pedal 2: Height DOWN

Pedal 3: Back UP

Pedal 4: Back DOWN

Pedal 5: Seat UP

Pedal 6: Seat DOWN


Pedal M1: *Memorizable position 1

Pedal M2: *Memorizable position 2

Pedal AUT: Automatic return from any position to the "End of treatment" position - (only for 3M version)

Pedal END: automatic return from any position to the zero position (flat and minimum height) - (only for 3M version)


***To memorize M1 (or M2) position:** set height, backrest and seat in the desired position, then press simultaneously **AUT + M1 (or M2)** button for about 5 seconds to preset memorized position. After memorized position is preset, press only button **M1 (or M2)** to move to memorized position. All controls are maintained action.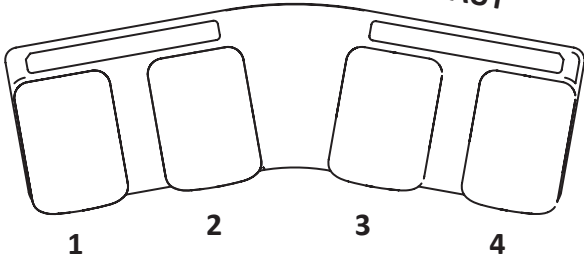

PODO MIX 2M PEDAL CONTROL


- Pedal 1** = Height UP
- Pedal 2** = Height DOWN
- Pedal 3** = Seat UP
- Pedal 4** = Seat DOWN
- Pedal 3+4 (AUT)**: Automatic return from any position to the "End of treatment" position

Once the treatment is ended, before the patient/client gets up, keep the chair to the initial position by pressing the AUT button/pedal.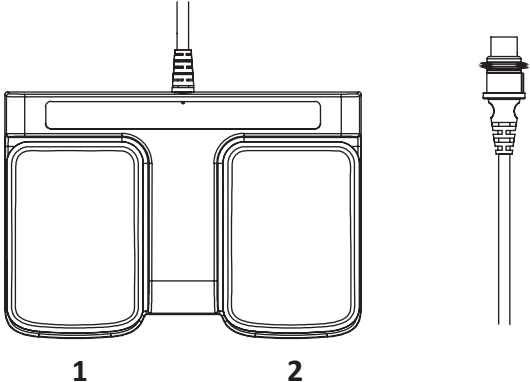

PODO MIX 1M PEDAL CONTROL

- Pedal 1** = Height UP
- Pedal 2** = Height DOWN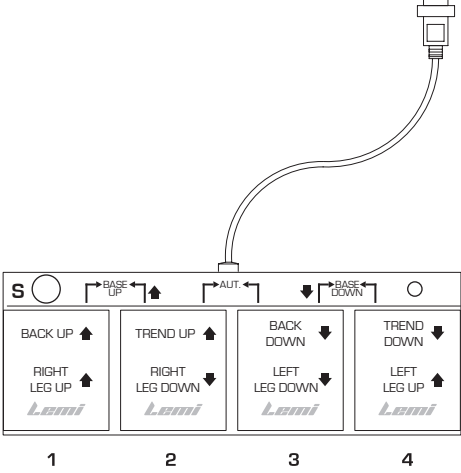

PODO 5 PEDAL CONTROL

- Pedal 1**: Back up
- Pedal 2**: Trendelenburg up
- Pedal 3**: Back down
- Pedal 4**: Trendelenburg down
- Pedal 1+2**: Chair up
- Pedal 3+4**: Chair down
- Pedal AUT (2+3)**: Automatic return to original position of the chair, to allow customer to get up or down easily


! To **ACTIVATE** the electric legs, press the **S BUTTON** (green LED on) **BEFORE** pressing any pedal.

- S (SELECT)**: Setting in motion of the pedal for operation of the legs (green LED on)
- Pedal 1**: Right Leg up
- Pedal 2**: Right Leg down
- Pedal 3**: Left leg down
- Pedal 4**: Left leg up
- Pedal 2+3**: Descent of both legs at the same time

! **ATTENTION!** BEFORE lowering completely the legs by the pedal or BEFORE using AUT control, be sure the legs have been fully shortened in order to avoid any crush.

7.4. Zak headrest adjustment

The large and comfortable Zak headrest can be adjusted manually using a practical knob located behind the backrest. By unscrewing it gently, the head can be easily removed, while for the inclination it is enough to gently push the head forward or backward with two hands.

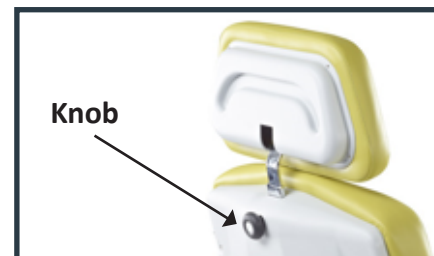
7.5. Manual back adjustment (Podo Mix 1-2M only)

Use the Back lever under the seat to adjust the inclination of the back rest. Always support with the handle the back during the movement.

7.6. Manual trendelenburg adjustment (Podo Mix 1M only)

To adjust the tilting inclination (Trendelenburg), rotate the tilting handle to OPEN position; once the correct position has been reached, rotate the handle to CLOSE position. Check the tilting handle is always closed before the patient sits on the chair.

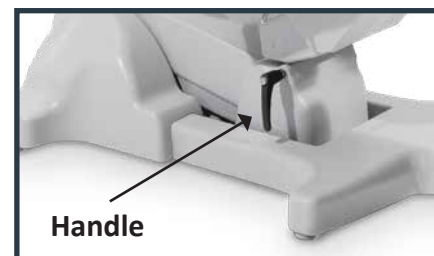ATTENTION! If the handle position is not comfortable for the operator, pull the handle, settle and release it.

7.7. Manual adjustment of the legs

- To STRETCH OUT the leg (Podo MIX 1-2 M only), turn the knob situated under the leg rest and pull the upholstery to reach the required position. Tighten the knob.
- To STRETCH OUT the AUTOMATIC LEG (Podo Mix 3M /Podo 5 only), it is enough to pull the leg to the required position.
- To OPEN the leg, hold the upholstery and push it right or left to the required position.
- To LOWER/LIFT the leg, hold the final part of the upholstery while using the lever for leg movement and push up or down.

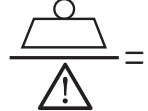
7.8. Chair Rotation

The upper part of the chair can rotate to 180° (90° left + 90° right). To rotate the chair, turn the rotation handle, fix the chair into the required position and block the rotation by tightening the handle clockwise.

8. USE OF THE CHAIR

 = **Max. patient weight**
230 kg (Podo Mix 3M)
200 kg (Podo Mix 3M/Podo 5)

 = **Safety load**
2300 N (Podo Mix 3M)
2000 N (Podo Mix 3M/Podo 5)

ATTENTION! Only one person can get up on the chair; the person must go up/down when the chair is in the basic position going up on the seat.

WARNING! Get on the chair always by sitting properly in the central part of the seat. Do not sit or lean on the side of the feet or of the head.

WARNING! The limbs of the patient must always be on the chair. The operator must always inform the patient when he moves the chair.


WARNING! Be careful to make sure that the chair or its electrical parts are not wet and that liquids become not stagnant on the chair itself; immediately wipe up any liquids – including physiological liquids of the patient – which may end up on the chair.

WARNING! If a patient is let alone for a short time, it is necessary to place the chair in the lowest position with respect to the walking floor, in order to reduce risk of damage in case of fall

To avoid shock hazard, do not connect or disconnect the power plug with wet hands. Do not use the chair barefoot.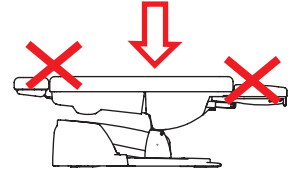

9. MAINTENANCE

The chair has been designed and constructed so as to reduce as much as possible the necessary maintenance and ensure proper functioning over the years without any problems. During any intervention on the chair, it is recommended to follow the instructions given in this section.

WARNING! All interventions on the chair must be carried out by competent and trained personnel on the methods to operate correctly and safety. Do not make any action on the chair moving, and always check that the plug connecting to power supply is disconnected; bring the power plug next to you during the execution of the intervention on the chair so you can check at any time the actual separation.

9.1. General Checks

Constantly check the general condition of the chair: the integrity of the protections, the electrical equipment and the condition of the pads.

WARNING! Do not use the chair if any component is found to be not perfectly intact or in service. In this case, put the chair out of service by disconnecting the power plug immediately and contact an Authorized Service Centre.

WARNING! Do not modify this chair without authorization of the manufacturer. If this chair is modified, appropriate inspection and testing must be conducted to ensure its continued safe use.

9.2. Extraordinary Maintenance

Any extraordinary maintenance must be performed by qualified personnel of the technical service Brusaferrri & C. or Service Centre authorized by the manufacturer.

Information about the Authorized Service Centre to which you can turn, can be obtained directly from the manufacturer Brusaferrri & C srl – phone 0374/363068-69 – mail: service@lemigroup.it

For a correct and quickly resolution of the problem, it is necessary to communicate the model and the serial number of the chair and the description of the defect, or what are the visible problems, and under what condition it occurs.

The broken chair, if under warranty, must be sent to the Service Centre together with the certificate of guarantee, transport charges must be paid by the purchaser.

WARNING! Any form of warranty expires if interventions are carried out on the chair by someone not authorized by Brusaferrri & C.

9.3. Authorized Personnel

Repairs and maintenance of the chair can be carried out exclusively by competent personnel, that is of Brusaferrri & C. or authorized by it. Information on the staff to contact for possible interventions can be obtained directly from Brusaferrri & C.

9.4. Cleaning

Each time you make a treatment, always cover the mattress with towels or disposable towels before the patient sit on the chair (See §7.2. Use of the mattress).

Pay particular attention to the jeans that can stain the covering without any possibility of being removed.

Clean the chair before each use, paying particular attention to remove all traces of physiological fluids of the previous patient. To clean the chair use a cloth and lukewarm water (if necessary diluted neutral detergent). If necessary, use a non-aggressive disinfectant, then wipe off with a clean dry cloth.

Do not use products such as alcohol, acetone, bleach, paint thinner, gasoline.

WARNING!

- Be careful not to spray liquids on the electrical equipments, pedal/push-button strip and on the protections of the chair.
- You must follow the handling directions given in the safety data sheets of the detergents used.
- While during the cleaning operation of the chair, inhaling powders are produced, use a proper protective device and air adequately the room.
- Use proper individual protection devices

10. IN CASE OF AN ACCIDENT

Any serious accident occurring in relation to this device must be immediately reported to the manufacturer and the competent authority of the member state in which the user and / or patient is established.

ATTENTION! Do not remove protections in any case; do not use the chair if the protections are damaged or not correctly fixed.

ATTENTION! Do not put limbs near to the moving parts of the chair.

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WARNING! Any form of warranty expires if they are used spare parts other than those originally fitted.

12. DISMANTLE

If the chair becomes inoperative or is not to be used for an extended period of time, it is recommended to remove it from service and remove the plug from the socket. All materials used to manufacture the chair are not toxic or hazardous to the health of operators, so they can be handled without special precautions. In case of disable of the chair you should perform a separation of different materials for subsequent reuse or differentiated elimination. All electrical and electronic products should be separately disposed, using specific collection facilities appointed by the Government or the Local Authorities.

13. DISPOSAL OF OBSOLETE EQUIPMENT

The correct disposal of the obsolete equipment will help to prevent potential negative effects on human health and the environment. For information, contact your Local Council or the Waste Disposal Service.

- **The dismantling of the chair at the end of its functional life must be done by qualified personnel trained about the correct way of handling – and if necessary of dismantling the chair.**
- **Use only suichair equipment and lift appliances in accordance with the National Regulations and Legislative Law in force.**
- **Having to dismantle the chair or to move it in another place, please contact the manufacturer Brusaferrì & C – Italy.**

IMPORTANT NOTICE HOW TO CLEAN THE ABS PLASTIC COVERING

It is forbidden to use the following products which could seriously damage the plastic covering, provoking cracks and the subsequent breaking:

- Degreasing solvent-based agents: e.g. ACETONE, DENATURED ETHYL ALCOHOL, TRICHLOROETHYLENE, FUEL, TURPENTINE...
- Watered-down acids (HYDROCLORIDRIC ACID, NITRIC ACID) or products containing watered-down acids
- Watered-down bases (CAUSTIC SODA, SODIUM HYPOCHLORITE, AMMONIA, BLEACH) or products containing weak bases
- Fluorinated hydrocarbons (e.g. LEAKS FROM REFRIGERANT GAS)
- Food-grade oils (e.g. OLIVE OIL, SEEDS OIL)
- Lubricating mineral-based oils

To CLEAN the coverings we recommend:

- Soap water
- Water and vinegar
- Non abrasive cream cleanser

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This publication or parts thereof may not be reproduced, stored in a storage machine, transmitted, transcribed, or translated into any language, usual or computer, in any form or by any means, electronic, mechanical, magnetic, optical, chemical, manual or otherwise, without the express written permission of the manufacturer, Brusaferrri & C. The manufacturer shall provide warranty with respect to the contents of this publications, with the express prohibition of the use of the same publication for any other purpose whatsoever, commercial, practical, technical-scientific, unrelated to the functioning of the equipment to which it relates for which this issue has been written. The warranty on the accuracy of the information contained herein is provided on condition that they are strictly observed by the user of the device, all the prescriptions contained in this documentation. Moreover Brusaferrri & C. reserved the right to revise this publication and to make changes in the content without obligation to notify any person or organization. These revised publications are available on request by Brusaferrri & C.

©2023 Brusaferrri & C. – All rights reserved.

Designed and manufactured by:

Lemi[®]
ITALIAN WELLNESS EQUIPMENT

Via Mara Maretti Soldi, 13
26011 Casalbuttano (CREMONA) ITALY
Phone +39-0374-363069 Fax +39-0374-363071
info@lemigroup.it service@lemigroup.it
www.lemigroup.it