SOSUL LINE / SOMED

ORIGINAL INSTRUCTIONS - REV. 3 / 2023

SOSUL 2000
SOSUL TOP 2-3 M

SOSUL

SOMED

INSTRUCTION FOR USE pag. 4

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EN

Congratulation for having chosen an equipment LEMI produced by Brusaferrri & C.

This chair has been designed and built using state of the art technologies and processes to guarantee the best reliability over time and ensure maximum safety.

For a correct and safe operation of the chair it is essential to carefully read and observe what is indicated in this manual. All documentation accompanying the chair – and in particular this manual – must be carefully preserved for future reference.

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Symbols meaning

Non-compliance of this procedure could cause damages to the table or its parts.

Non-compliance of this procedure could cause health damages or injury to the operator or people near the table

1. INTRODUCTION

The successful and safe operation of the table is guaranteed only if it is used in accordance with the specifications in this manual and, in general, in the accompanying documentation; therefore it is essential to carefully read and keep all the relative documentation. Although the table is equipped with active and passive safety features, not all risks can be avoided due to its incorrect use. You must always make sure that all users have fully understood the rules of use; the manufacturer declines any responsibility for damage to persons or property resulting from improper use of the device. Do not remove or damage the labels, notices and warnings placed on side of the table. In case you need to restore them, please contact the manufacturer. The manufacturer Brusaferrri & C., declines any and every responsibility for failure to observe the rules of safety and prevention described in the various sections of this manual and for any damage caused by improper use. All works on the table (maintenance, adjustments, repairs, cleaning) must be carried out by suitably trained personnel and in accordance with what is indicated in this manual.

1.1. Intended use and durability

The SOSUL LINE tables have been designed and built to perform cosmetic face and body treatments, massages, SPA treatments, diagnostics, physiotherapy, rehabilitation. Somed has been designed to perform diagnostic examinations, physiotherapy, rehabilitation, reflexology and massage.

WARNING! It is forbidden any use of the chair other than that described in this manual. Brusaferrri & C. declines any responsibility for malfunctions or damages to persons or property due to uses other than those specified in this manual.

The chair should be used only by personnel informed about its use mode, operation and safety equipment.

1.2. Preservation of the manual

This operating manual should be kept for future reference. We recommend:

Keep the manual in a place accessible and known to all operators, protected from moisture, heat and direct sun beams. Use the manual paying attention not to damage any part of its content: do not remove, tear or modify any part of the manual.

In case of sale or transfer of the chair to a third party, this manual and its annexes should be delivered intact to the new users.

WARNING! Please read this operation manual before using the chair. Anyone who uses the chair should be properly informed on the parts of this manual relevant to the operations to be performed.

1.3. Labelling of the chair

The chair is provided with labels with the following data, in conformity with the current rules (see § 14. Label Legend)

WARNING! Do not to remove the labels placed on the machine; labels must be preserved well fixed, intact and easy to read.

WARNING! The data present on each label, must be reported when requiring assistance or spare parts.

1.4. Warning messages

The chair is provided with signs showing phrases or symbols; pay attention to the warnings on the equipment.

The electrocution risk warning sign is located next to the electric supply input plug.

The yellow label warning "DON'T PUT YOUR FOOT ON – DON'T PUT OBJECTS ON" is located next to moving parts: it warns against putting any object or limb in the chair movement area in order to avoid crushing damages.

2. TRANSPORT AND INSTALLATION

2.1. Handling and removal from the packaging

The table can be raised and moved manually by two or four people, taking it exclusively from the 4 support feet.

The weight of the table is equal to 77 kg.

The table is shipped packed in a carton box. Open the box so you can freely raise the table; remove it from the packaging, taking it from the 4 feet and pull it upwards.

ATTENTION!

- Handling and moving operations have to be done by qualified and instructed personnel.
- It is strictly forbidden to remove the table by lifting it from the top (mattress).
- Do not use knives or box cutters to open the box so as not to risk damaging the table and in particular the pudding.
- Before using the table, clean it entirely according to the procedure shown at § 9.4. Cleaning

2.2. Flooring

The floor on which the table will be placed must be solid, smooth, horizontal and adequate to support its weight and ensure the stability.

2.3. Lighting

The user must ensure that the lighting of the work areas is sufficient to allow a good view of the operations and all elements of the table; the level of lighting should be in accordance with the UNI EN 12464-1; in particular should be avoided shadows and annoying glare.

2.4. Positioning

The position of the table compared to fixed obstacles must be such as to allow the easy passage for the use and maintenance; in particular, the minimum space to allow the passage of persons shall be equal to 650 mm (in accordance with UNI EN 349:2008). The minimum distance between the completely lowered back and fixed obstacle has to be 500 mm, in order to avoid the body to be crashed by the table.

2.5. Levelling

The table is provided with 4 adjustable feet that enable to level it perfectly and to compensate any failing of the flooring. The regulation can be done by screwing or unscrewing the feet.

2.6. Power supply

The system of power supply must have voltage and frequency equal to the rated voltage and frequency of the power supply of the device and be appropriate to the installed capacity, as indicated on the identification plate of the table.

Voltage	100 V / 240 V
Frequency	50/60 Hz
Power / Absorption	1,5 Amax (0,6 A) Sosul - Sosul 2000 - Somed / 4 Amax (1,7 A) Sosul Top 2M/3M
Power supply	Alternate Current (AC) One Phase L+N
Duty cycle	2 min / 18 min at 10% maximum
IP Protection	IP54

Centralina classe II

The power supply must be supplied to the unit by a line:

- Protected against overcurrent by an automatic circuit breaker (circuit breaker) or fuses, properly sized; the operating current of the protections must be as low as possible taking into consideration the maximum absorption of the device;
- Equipped with a device sensitive to insulation failure (differential) with operating current low enough; this device must be positioned downstream of the overcurrent protective device (or be integrated with it);
- Having cables with section suitable for the power absorbed by the appliance.

To avoid the risk of electric shock, this appliance **must only be connected to a power supply with grounding.**

2.7. Connections

START (ON): plug the power cable into a suitable electric socket. If the outlet and the plug of the chair are not compatible, ask qualified personnel to replace it with a suitable one.

STOP (OFF): Remove the plug from the electric socket.

ATTENTION! Pull out by grabbing THE PLUG, NOT THE CORD

WARNING! It is strongly recommended to insert the plug into a single outlet exclusively dedicated to the device. We do not recommend the use of multiple sockets (power strip), adapters and extension cables. The cable must not be bent.

WARNING! Before plugging in the cable, make sure that it is not twisted or bended and make sure that the protective sheath is undamaged.

WARNING! The chair must be positioned in order to ease the removal of the electric plug.

WARNING! To avoid the risk of electrical shock, this chair must exclusively be plugged in power supply grids with grounding.

3. ENVIRONMENTAL WORKING CONDITIONS AND STORAGE

The chair has been designed and built to operate in a closed and dry room. The operating environmental conditions are:

- temperature: 5° ÷ 40° C 
- humidity: 30% ÷ 75% without condensation. 

If the environmental conditions are particularly critical it is advisable to provide the room with an adequate air conditioning system to bring the humidity and temperature values within acceptable limits.

WARNING! Do not expose the chair to an excessively humid environment. If the chair gets wet, remove the plug from the outlet and have it checked by an Authorized Service Centre.

WARNING! The chair was not conceived to be used in settings with explosive atmospheres, in presence of thin powders or corrosive gases, acids, corrosive agents or salt.

WARNING! It is recommended to check the adequacy of the electromagnetic performance of the devices adjacent to the chair, making sure that it continues to operate normally.

- Not intended to be used in oxygen rich environments.

- Not intended to be used with flammable anesthetics or in conjunction with flammable agents.

STORAGE! For the correct functioning of the electrical equipment the chair must be stored at a temperature between -25° C and +55° C, and only for short periods not exceeding 24 hours, to +70° C. The chair has no special requirements for storage, provided that it takes place in a closed and dry rooms; this condition should also be ensured during all phases of transport.

4. PRELIMINARY INSPECTIONS

Before using the chair, please check that:

- the equipment does not present any damage arising from transport or handling; if needed, please contact the manufacturer Brusaferrri & C to solve such inconveniences;
- all packing (carton box, foam, plastic) have been removed;
- the voltage of your cabin is the same as the voltage indicated in section 2.6 Power Supply;
- all control devices (electrical or manual) work correctly

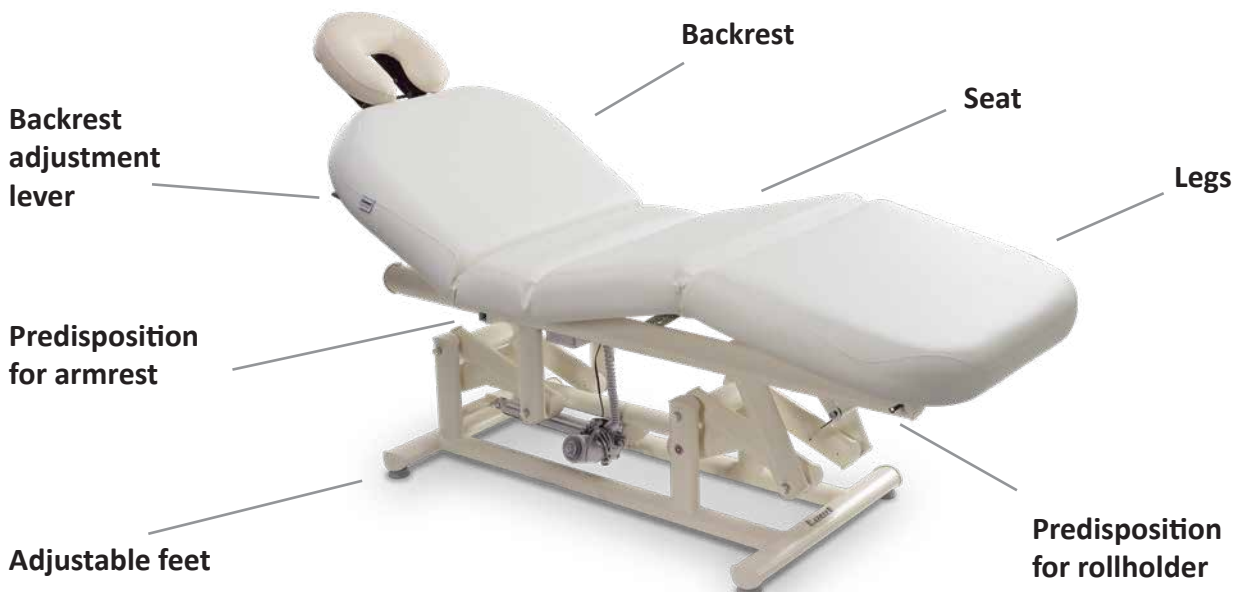
- For the installation, please contact qualified personnel only.
- Before the start-up of the chair, run a few idling cycles test, such tests must be performed in safety conditions by qualified personnel.

5. DESCRIPTION OF THE CHAIR


SOSUL

SOSUL 2000 / SOSUL TOP 2 - 3M

SOMED

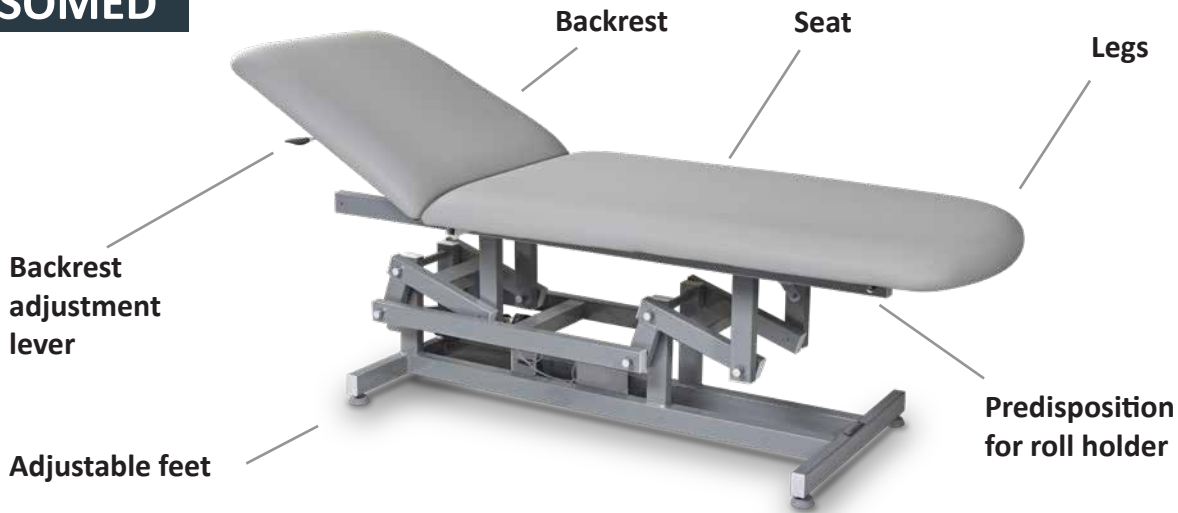6. OPTION ON DEMAND

6.1. Heating

- OFF = heating off (led: ON)
- ON = heating on (led: ON + LOW)
- LOW = low temperature 40° (led: ON + LOW)
- HIGH = higher temperature 47° (led: ON + HIGH)
- 2H = heating is on for 2 hours (LOW or HIGH), then turn off automatically.

When activated, the led blinks till the end of the second hour. You can stop heating in any time pressing OFF button.

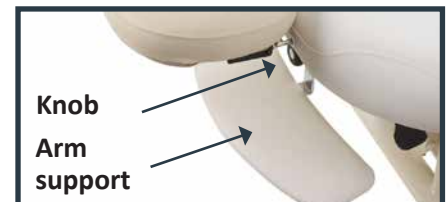
ATTENTION! It is absolutely necessary **TURN OFF THE HEATING (both led off) WHEN THE TABLE IS NOT USED AND AT THE END OF EACH WORKING DAY.**

The heating works with the pressure of the body, so we do not feel any sensation of heat if it is not stressed by the weight of a body. It is recommended to always leave a weight laid (for example towels or magazines) to maintain the minimum temperature.

6.2. Arm support (Sosul Top 2-3M - Sosul 2000)

To install/remove the arm support, unscrew the knob placed under the backrest.

6.3. Wheels

The 4 wheels with brake (∅ 50mm) are installed in place of the feet.

WARNING! ALWAYS block the brake of each wheel before the client sits on the table.

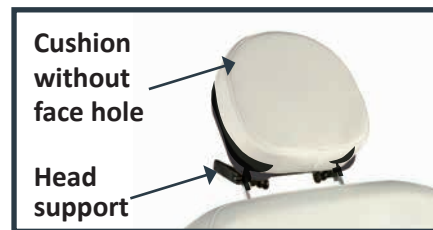
6.4. Rollholder

The table can be equipped with a rollholder, which is installed by using specific strings fixed to two screws on the backrest.


6.5. Cushion without face hole (Sosul Top 2-3M - Sosul 2000)

Remove the face hole headrest cushion from the head support (it is fixed by Velcro®) and place the cushion without hole.

6.6. Armrests (SOSUL LINE)

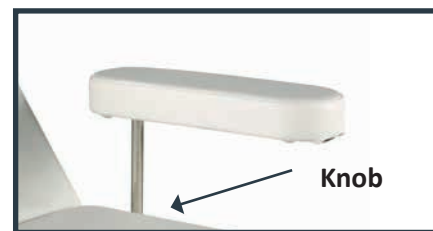To assemble the armrests, place them in the predisposed support and screw the knobs in the clutches situated on both sides near the backrest.

WARNING! These armrest were only meant to support the arm. **It is strictly forbidden to seat on the armrests.** Any damage caused by improper use, would not be considered under warranty.

7. INSTRUCTIONS FOR USE

7.1. Operator

The chair is controlled by an operator through the foot pedal (or the handset control); the operator must position themselves so as to have complete view of the controlled parts and of the patient.

WARNING! During the functioning of the chair the operator should always remain near the controls; it is forbidden to leave the chair unattended during its operation.

Operators should pay attention to avoid creating potentially dangerous situations for their own or other's safety.

All people working on the chair must be properly trained on the correct methods of work and informed about its proper functioning. They also need to be aware of the parts of this documentation relevant to their work.

Operators must not wear clothes with drapes that could get caught by moving part of the chair. Proper clothing consists of close-fitting garments, especially on the upper limbs.

Both the operator and the patient must not wear rings, necklaces, bracelets, wristwatches, ties or other objects that could get caught by moving parts of the machinery.

In case of allergy, cover the chair with a sheet.

The operator must always have under control all the moving parts and the patient.

7.2. Use of the mattress

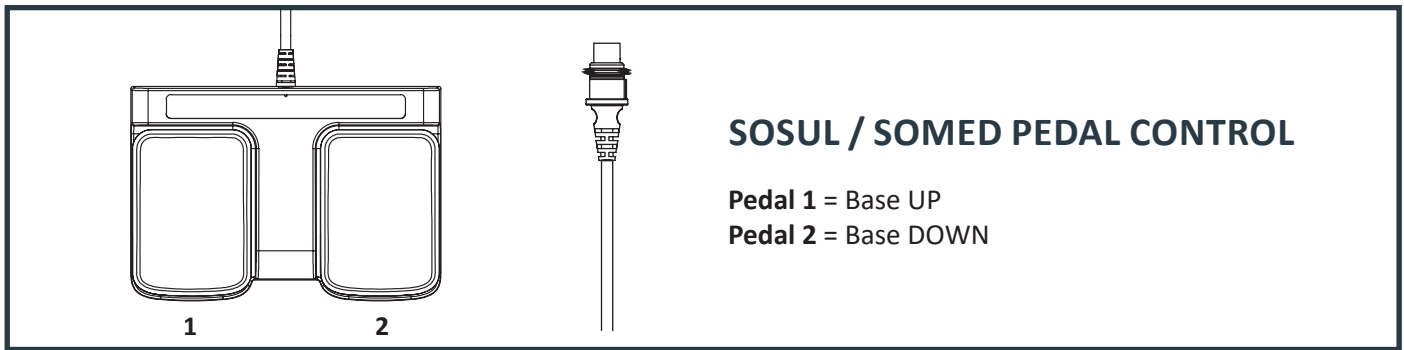
IMPORTANT! It is mandatory to cover and protect the mattress with disposable paper covers or washable sponge covers (better if sponge + PVC) for its correct preservation and for hygienic purposes. The upholstery could be damaged by the normal heat of the skin or by a prolonged use of essential oils or cosmetic product. Any replacement related to damages due to misuse won't be considered under warranty.

7.3. Control devices

The chair can be controlled by a foot pedal control with cable or a wireless pedal control. All the control devices are of the hold-to-run type: the chair movement will stop as soon as the operator releases the pedal, except for the AUT position. This movement can be stopped by pressing any pedal. A single bed can be connected to max. 6 wireless control devices.


The operator can move the examination chair only through the control devices.


SOSUL / SOMED PEDAL CONTROL

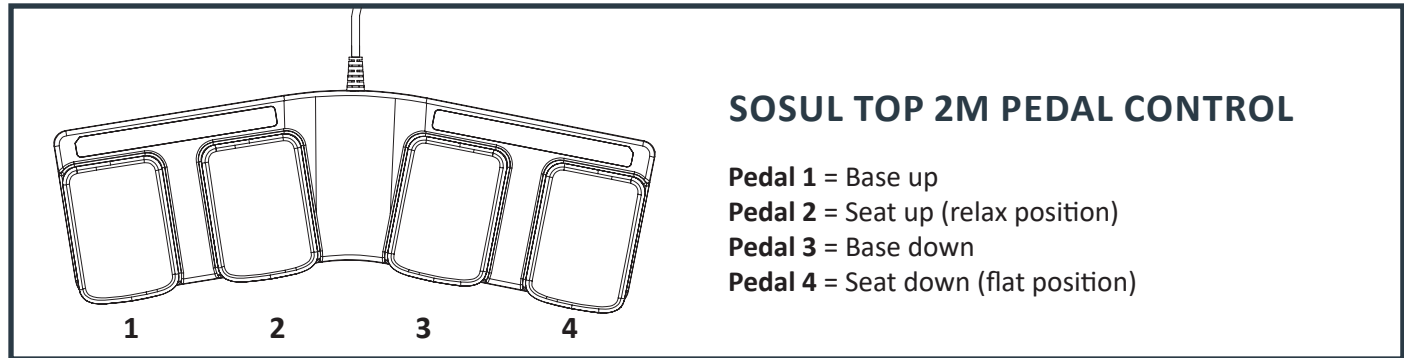- Pedal 1** = Base UP
- Pedal 2** = Base DOWN

SOSUL TOP 2M PEDAL CONTROL

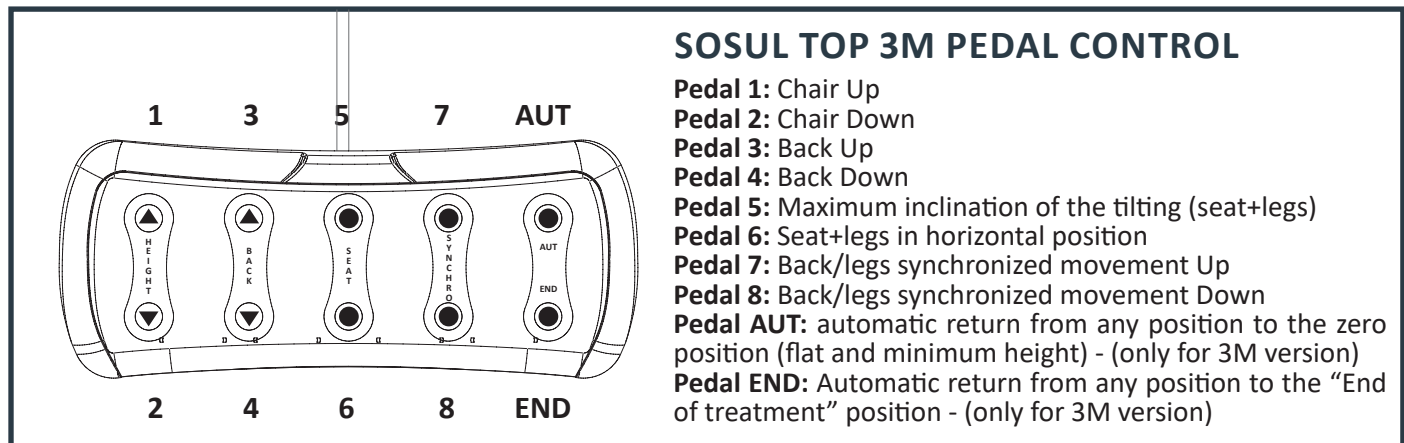- Pedal 1** = Base up
- Pedal 2** = Seat up (relax position)
- Pedal 3** = Base down
- Pedal 4** = Seat down (flat position)

SOSUL TOP 3M PEDAL CONTROL

- Pedal 1:** Chair Up
- Pedal 2:** Chair Down
- Pedal 3:** Back Up
- Pedal 4:** Back Down
- Pedal 5:** Maximum inclination of the tilting (seat+legs)
- Pedal 6:** Seat+legs in horizontal position
- Pedal 7:** Back/legs synchronized movement Up
- Pedal 8:** Back/legs synchronized movement Down
- Pedal AUT:** automatic return from any position to the zero position (flat and minimum height) - (only for 3M version)
- Pedal END:** Automatic return from any position to the "End of treatment" position - (only for 3M version)

Once the treatment is ended, before the patient/client gets up, keep the chair to the initial position by pressing the AUT button/pedal.

7.4. Zak headrest adjustment (Sosul)

The large and comfortable Zak headrest of Sosul can be adjusted manually using a practical knob located behind the backrest. By unscrewing it gently, the head can be easily removed, while for the inclination it is enough to gently push the head forward or backward with two hands.

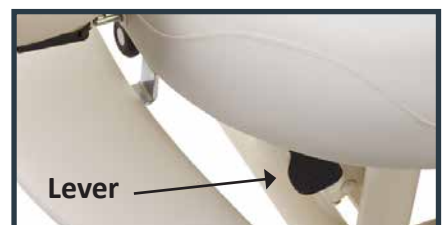
7.5. Head adjustment (Sosul 2000 - Sosul Top 2/3M)

The new padding is fixed on the new head support made of a super resistant, durable and hygienical plastic material. Using the side handle, it is possible to adjust the inclination of the head support and padding, as well as fix the distance from the mattress.


7.6. Manual back adjustment (Sosul/Sosul 2000/Sosul Top 2M/Somed)

To raise the back rest, hold it with one hand and press the lever with the other hand. To lower the back rest (horizontal position), press the lever and push down the back.


7.7. Manual leg adjustment

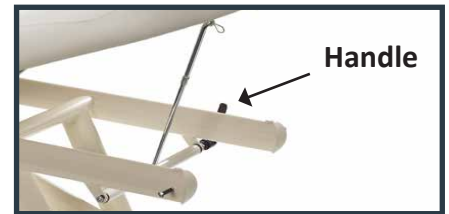To raise the leg (final part of the mattress), open the handle and pull upwards the leg mattress. Fix into the required position by closing the handle.


7.8. Manual seat/leg adjustment (relax position – Sosul 2000)

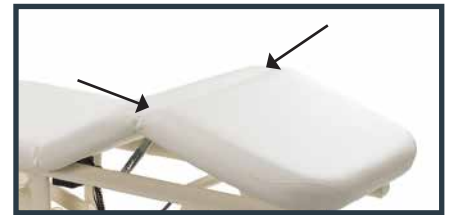5 different inclinations may be obtained by seizing the ending part of legs mattress and pulling it gradually upwards. To lower the legs, pull again the legs mattress upwards to the limit and follow the movement downwards.

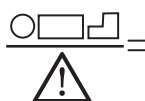8. USE OF THE CHAIR

 = **Max. patient weight**
150 kg (Somed) / 200 kg (Sosul line)

 = **Safety load**
1500 N (Somed) / 2000 N (Sosul line)

ATTENTION! The chair can only accommodate one person at a time; the person must sit/get up when the chair is in the original position.

WARNING! Always get on the chair by sitting in the central part of the seat. Do not sit or lean on the feet or head sections.

WARNING! The patient must always keep the limbs on the chair. The operator must always warn the patient when he moves the chair.

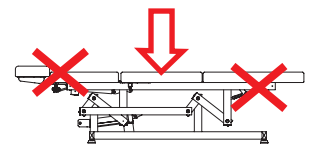
WARNING! Always make sure that the chair or its electrical parts are not wet and that liquids do not stagnate on it; in case the chair gets wet, immediately wipe up any liquids – including physiological liquids of the patient.

WARNING! If a patient is left alone for a short time, it is necessary to regulate the chair in the lowest position, in order to reduce the risk of damage in case of fall.

To avoid shock hazard, do not connect or disconnect the socket with wet hands. Do not use the chair barefoot.

9. MAINTENANCE

The chair has been designed and built so as to reduce as much as possible the necessary maintenance and ensure proper functioning over the years without any problems. During any intervention on the chair, it is recommended to follow the instructions given in this section.

WARNING! All interventions on the chair must be carried out by competent personnel trained on the methods to operate correctly and safety. Do not intervene on the chair when moving, and always check that it is not connected to power supply; keep the socket next to you during the intervention on the chair so you can check at any time the actual disconnection.

9.1. General Checks

Constantly check the general condition of the chair: the integrity of the protections, the electrical equipment and the condition of the padding.

WARNING! Do not use the chair if any component is not perfectly intact or working. In this case, immediately turn off the chair by disconnecting the power plug and contact an Authorized Service Centre.

WARNING! Do not modify this chair without the manufacturer authorization. If this chair is modified, appropriate inspection and testing must be conducted to ensure its continued safe use.

9.2. Extraordinary Maintenance

Any extraordinary maintenance operation must be performed by qualified personnel of the technical service Brusaferrri & C. or Service Centres authorized by the manufacturer. Information about the Authorized Service Centre to which you can address, can be obtained directly from the manufacturer Brusaferrri & C srl – phone 0374/363068-69 – mail: service@lemigroup.it. For a correct and quick resolution of the problem, it is necessary to communicate the model and the serial number of the chair, the description of the problem, and under what condition it occurs. The broken chair, if under warranty, must be sent to the Service Centre together with the warranty certificate, transport charges must be paid by the purchaser.

WARNING! Any warranty expires if any intervention on the chair is performed by someone who is not authorized by Brusaferrri & C.

9.3. Authorized Personnel

Any extraordinary maintenance intervention of the chair can be carried out exclusively by competent personnel, from or authorized by Brusaferrri & C. Information on the staff to contact for possible interventions can be obtained directly from Brusaferrri & C. (see §9.2. Extraordinary Maintenance).

9.4. Cleaning

Each time you make a treatment, always cover the mattress with towels or disposable towels before the patient sits on the chair (See §7.2. Use of the mattress).

Pay particular attention to the jeans, that can stain the upholstery without any possibility of being removed.

Clean the chair before each use, paying particular attention to remove all traces of physiological fluids of the previous patient. To clean the chair use a cloth and lukewarm water (if necessary diluted neutral detergent). If necessary, use a non-aggressive disinfectant, then wipe off with a clean dry cloth.

Do not use products such as alcohol, acetone, bleach, solvents, gasoline.

WARNING!

- Be careful not to spray liquids on the electrical equipments, pedal/push-button strip and on the protections of the chair.
- You must follow the handling directions given in the cleaning products safety data sheets.
- In case inhalable dust is produced during the cleaning operation of the chair, use a proper protective device and ventilate adequately the room.
- Use proper individual protection devices

10. IN CASE OF ACCIDENT

Any serious accident occurring in relation to this device must be immediately reported to the manufacturer and the competent authority of the country state in which the user and / or patient is established.

Do not, under any circumstances, remove protections; do not use the chair if the protections are damaged or not correctly fixed.

Do not put your limbs close to the moving parts of the chair.

11. SPARE PARTS

All the spare parts must be requested to Brusaferrri & C. that will supply them directly or will provide information on where they can be found. Please note that any extraordinary maintenance must be performed by personnel of Brusaferrri & C. or authorized by it.

WARNING! Any form of warranty expires if spare parts other than those originally installed are used.

12. DISMANTLE

If the chair becomes inoperative or is not to be used for an extended period of time, it is recommended to put it out of service and remove the plug from the socket. The chair is made of steel, wood and plastics, therefore it can mostly be recycled. All the materials used to manufacture the chair are not toxic or hazardous to the health of operators, so they can be handled without special precautions. In case the chair is put out of service, the different materials should be separated for reuse or disposal. All electrical and electronic parts should be separately disposed, using specific collection facilities appointed by the Government or the Local Authorities.

13. DISPOSAL OF OBSOLETE EQUIPMENT

The correct disposal of the obsolete equipment will help to prevent potential negative effects on human health and the environment. For information, contact your Local Council or the Waste Disposal Service.

- The dismantling of the chair at the end of its service life must be done by qualified personnel trained about the correct way of handling – and if necessary of dismantling the chair.
- Use only suitable equipment and lifting devices in accordance with the National Regulations and Legislative Law in force.
- Please contact the manufacturer Brusaferrri & C – Italy to dismantle or move the chair to a different location.

IMPORTANT NOTICE HOW TO CLEAN THE ABS PLASTIC COVERING

It is forbidden to use the following products which could seriously damage the plastic covering, causing cracks and the subsequent breaking:

- Degreasing solvent-based agents: e.g. ACETONE, DENATURED ETHYL ALCOHOL, TRICHLOROETHYLENE, GASOLINE, TURPENTINE...
- Acids diluted in water (HYDROCLORIDRIC ACID, NITRIC ACID) or products containing acids diluted in water
- Bases diluted in water (CAUSTIC SODA, SODIUM HYPOCHLORITE, AMMONIA, BLEACH) or products containing weak bases
- Fluorinated hydrocarbons (e.g. LEAKS FROM REFRIGERANT GAS)
- Food oils (e.g. OLIVE OIL, SEEDS OIL)
- Lubricating mineral-based oils

To CLEAN the coverings we recommend:

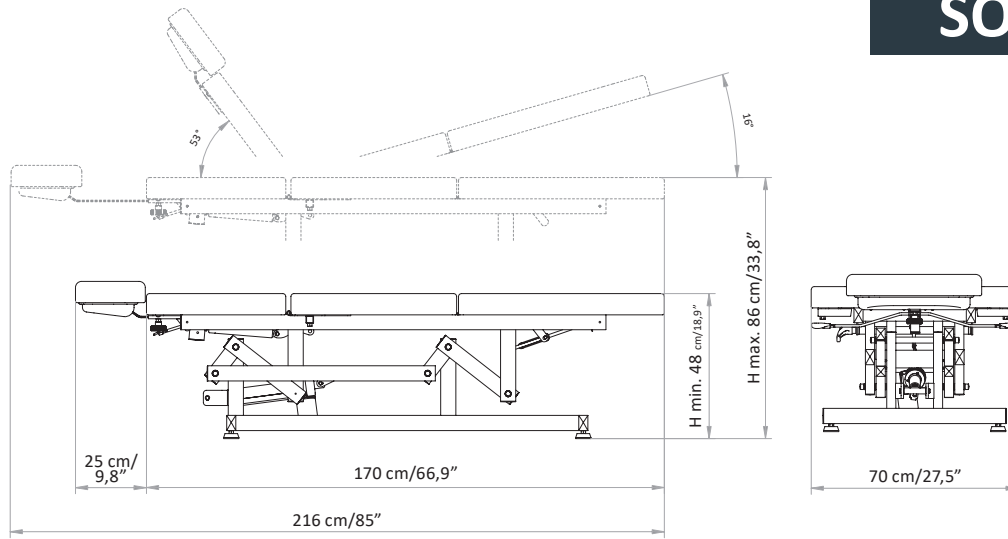
- Soap water
- Water and vinegar
- Non abrasive cream cleanser

Always rinse with clean water and dry with a soft, not abrasive cloth.

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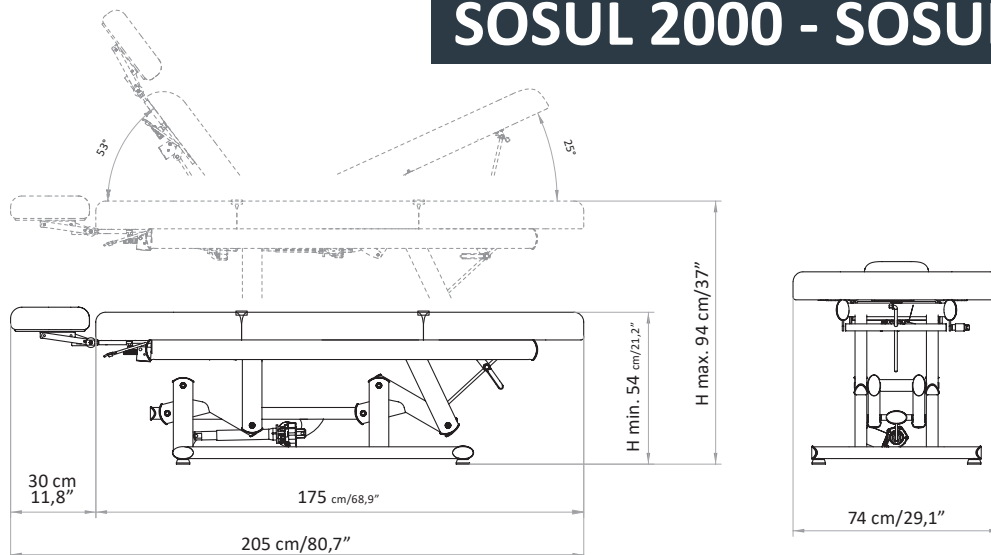
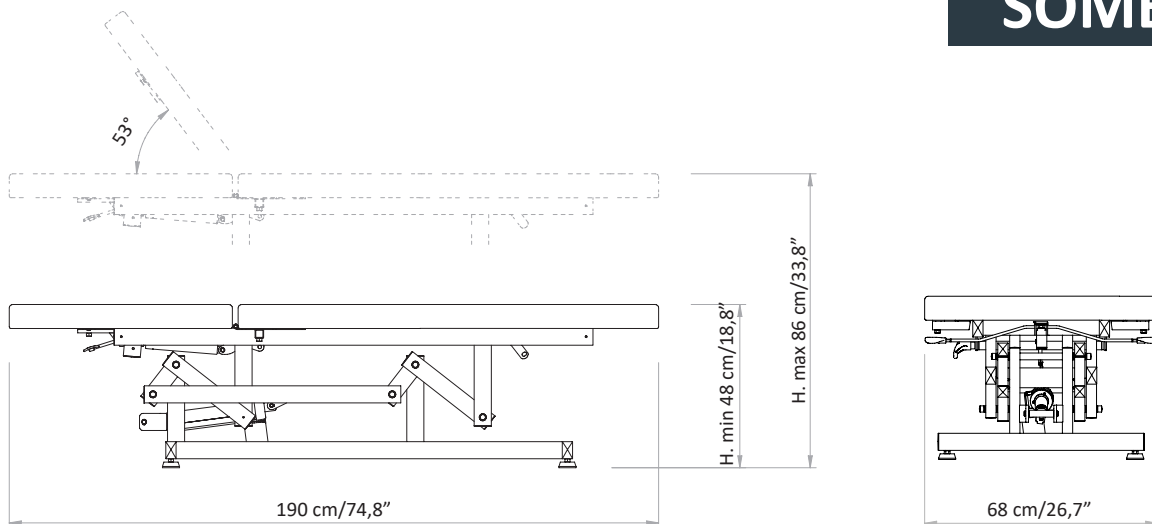
Operators must not wear clothes with drapes that could get caught by moving part of the chair. Proper clothing consists of close-fitting garments, especially on the upper limbs. Both the operator and the patient must not wear rings, necklaces, bracelets, wristwatches, ties or other objects that could get caught by moving parts of the machinery.

In case of allergy, cover the chair with a sheet. The operator must always have under control all the moving parts and the patient.

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This publication or parts thereof may not be reproduced, stored in a storage machine, transmitted, transcribed, or translated into any language, usual or computer, in any form or by any means, electronic, mechanical, magnetic, optical, chemical, manual or otherwise, without the express written permission of the manufacturer, Brusaferrri & C. The manufacturer shall provide warranty with respect to the contents of this publications, with the express prohibition of the use of the same publication for any other purpose whatsoever, commercial, practical, technical-scientific, unrelated to the functioning of the equipment to which it relates for which this issue has been written. The warranty on the accuracy of the information contained herein is provided on condition that they are strictly observed by the user of the device, all the prescriptions contained in this documentation. Moreover Brusaferrri & C. reserved the right to revise this publication and to make changes in the content without obligation to notify any person or organization. These revised publications are available on request by Brusaferrri & C.

©2023 Brusaferrri & C. – All rights reserved.

Designed and manufactured by:

Lemi[®]
ITALIAN WELLNESS EQUIPMENT

Via Mara Maretti Soldi, 13
26011 Casalbuttano (CREMONA) ITALY
Phone +39-0374-363069 Fax +39-0374-363071
info@lemigroup.it service@lemigroup.it
www.lemigroup.it